

HOW TO GUIDE

SilhouetteStar / SilhouetteConnect Imaging



Silhouette's flexibility allows for a variety of workflows. The following is one example.

PATIENT DETAILS

Step 1: For adding a wound assessment to an existing patient; from the home screen, use the search bar to find the patient.

OR

Step 2: To create a new patient. Tap **Create Patient** fill in the text boxes and save.

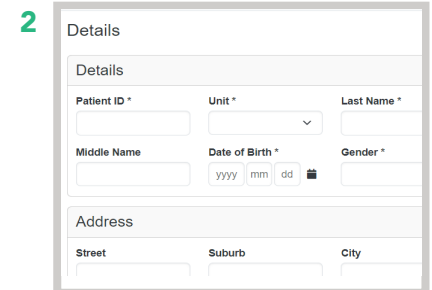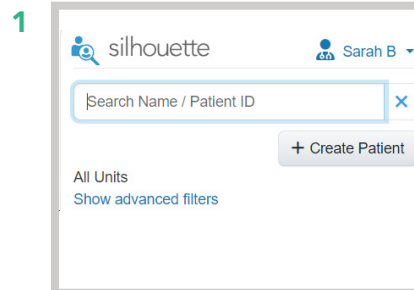

TO CREATE A WOUND ASSESSMENT

For creating a new wound



Step 3a: Tap **+ Wound**

OR

Step 3b: Tap **+ Add Assessment**

then **Wound Assessment**

then **+ Create New Wound**

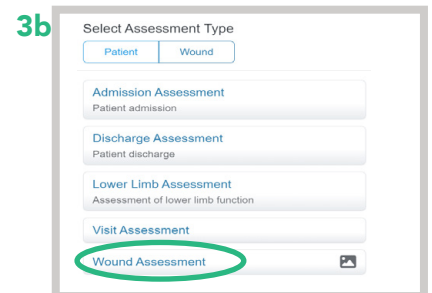
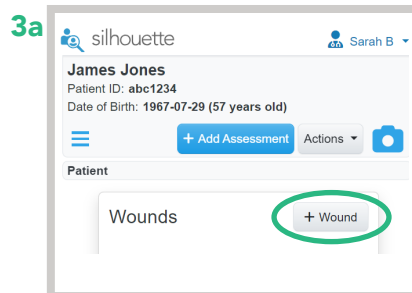


Follow the prompts to determine wound location and tap **✓ Save**

For adding an assessment to a known wound already created


Step 4: Tap on the wound label you wish to add an assessment to.

Step 5: Tap **+ Wound Assessment**

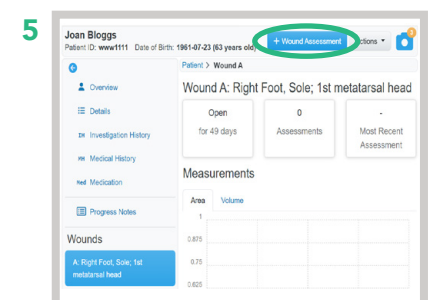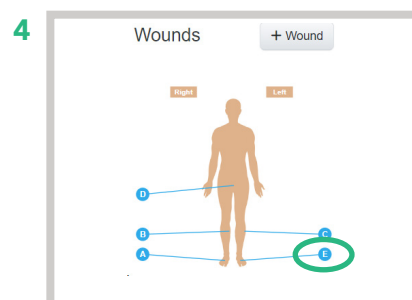

TO TAKE AN IMAGE

Step 6: Tap **+** and you will be ready to take an image.

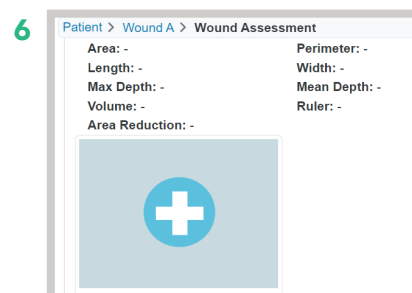


Step 7: Ensure SilhouetteStar is connected to your computer via the USB cable.

Step 8: Holding SilhouetteStar press the button and laser lights will show. When the laser lights cross in the wound, keep the SilhouetteStar still and press the button to take the image.


Step 9: The image will appear in the portfolio with a blue background. Select the image.


Step 10: Tap the wound image.

Step 11: The stylus can be turned on or off on the tool bar. The on-screen stylus is not required on a laptop with a mouse or touchpad and can be turned off.

Step 12: Select the  tracing tool.

Step 13: Trace around the perimeter of the wound or use the auto boundary detection tool.

Step 14: Once happy with boundary, tap .

Step 15: Scroll down the page and complete the wound assessment documentation, then tap .

