

# HOW TO GUIDE

## Silhouette Tracing Tool - Mobile Devices

### OVERVIEW

Tap mode on banner.



As you tap to change the function the icon on the banner will show the mode.




**Tool** controls include: tracing outlines, drawing rulers, editing, and marking a wound as healed.




**On-Screen Stylus** is used on touch screens to add and edit overlays on the image.

### Tool Dropdown Menu

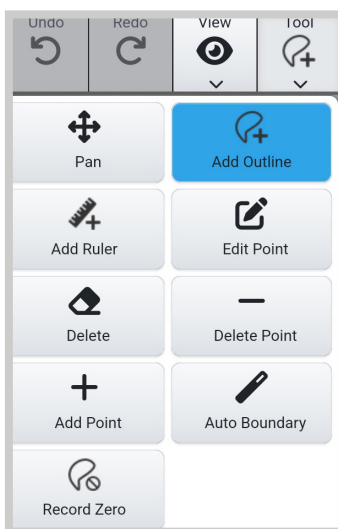

**View** the image with and without the boundary to allow the user clear site of wound margin. Rulers can be visible or hidden.

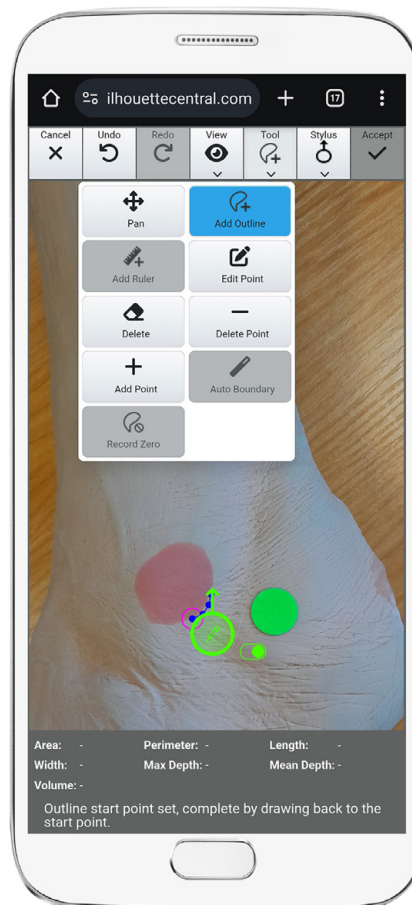


### On-Screen Stylus Dropdown Menu



**Redo** the last task.



**Undo** the last task.

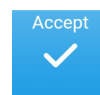

**Accept** once task is completed.



**Cancel** will remove the tracing and take you back to the image



Zoom in and out using the **Pan** tool and pinch gestures with your fingers on the screen.

OR



Use the appropriate magnifying glass on the View dropdown menu.

Use the **Pan** tool to move the image when it is zoomed.


## ON-SCREEN STYLUS



Select the preferred on-screen stylus option.

Slide to change size of on-screen stylus.


Alternatively, turn the on-screen stylus off and use a physical stylus.

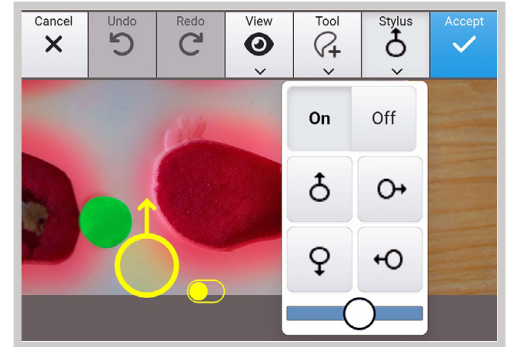

## TOOL - ADD OUTLINE





From Tool menu, select **Add Outline**.

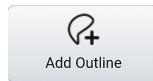

**Continuous tracing:** Click on the toggle to enable continuous draw mode. The on-screen stylus will turn green. Trace around wound.

**Dot point connecting:** Tap with on-screen stylus to create a point. Continue dropping points around the wound to complete the outline.

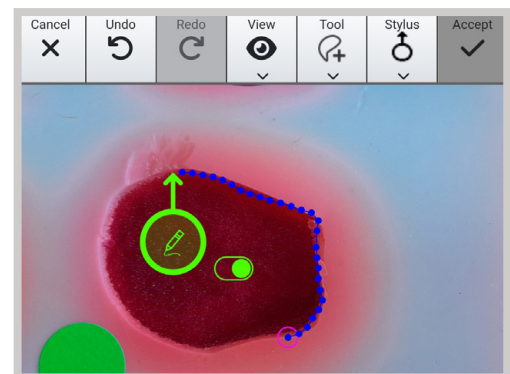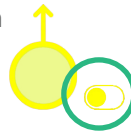


## TOOL - AUTO BOUNDARY



From Tool menu, select **Auto Boundary**.



The outline of the wound will be automatically detected and display. Check and edit if necessary.

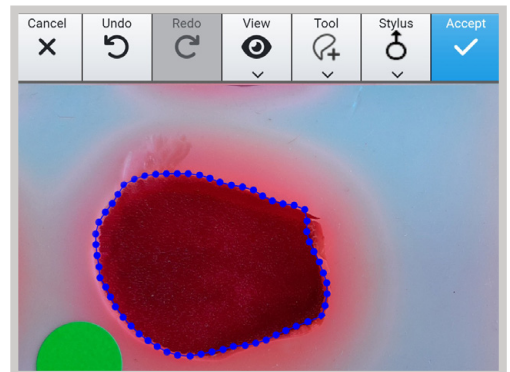


## TOOL - EDIT POINT



From Tool menu, select **Edit Point**.




Move the stylus to select the point you wish to edit, it will turn pink. Tap the on-screen stylus so it changes to green. The point can now be dragged to the desired location. Tap the on-screen stylus to set the new position.

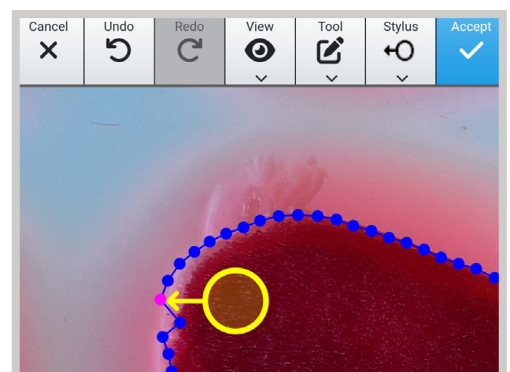


## TOOL - DELETE POINT



From Tool menu, select **Delete Point**.

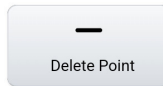


Move the stylus to select the point you wish to delete, it will turn pink. Tap on-screen stylus to delete it.

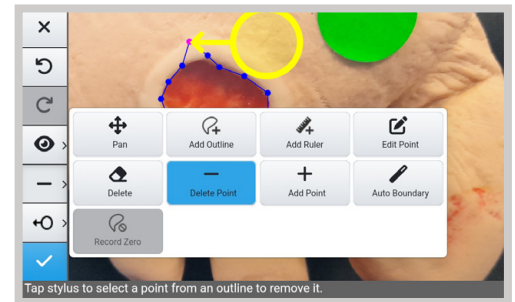

## TOOL - ADD RULER




From Tool menu, select **Add Ruler**.

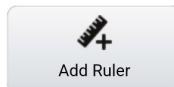

Move yellow on-screen stylus to where you wish to add the start point and tap. The stylus will change to green.


As you move the on-screen stylus to the location you wish to measure to, a line will show the ruler. Select the end point by tapping. A ruler will appear with the distance measurement.

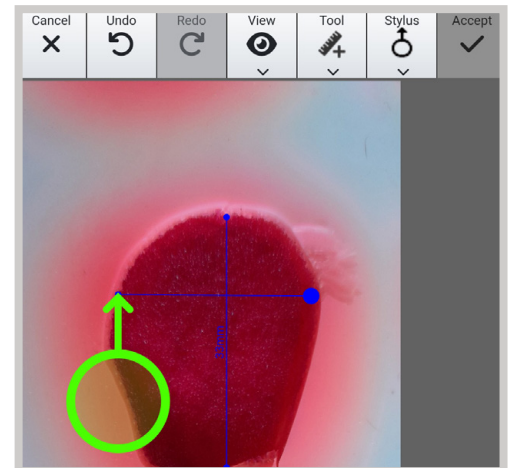

## TOOL - RULER TO DOCUMENT TUNNELING




To record undermining, measure with a calibrated probe and draw a series of dots on the skin following the undermining.

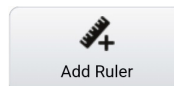

When tracing the wound in Silhouette, add a ruler, or a series of rulers using the drawing tool.

The wound can be traced in the normal way using Silhouette.

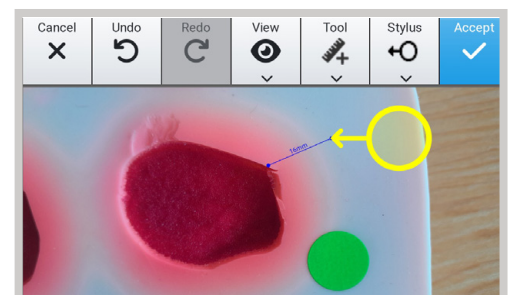

## TOOL - RECORD ZERO



This is for when you wish to record the wound site as a healed wound.

