# aranzmedical

# Silhouette v4.2 Administration User's Guide

Note: This system is developed for use only by qualified medical professionals trained and experienced in its use. Do not use the system before reading and understanding this user's guide.

This document has been prepared by ARANZ Medical Limited for its customers. The content of this document is confidential. It may be reproduced only with written permission from ARANZ Medical Limited. Specifications contained herein are subject to change, and these changes will be reported in subsequent revisions or editions. The device described in this document cannot substitute for the knowledge, skill and experience of the competent medical personnel who are its intended users. Its use as such a substitute is prohibited.

## Copyright © 2018 ARANZ Medical Limited

All rights reserved. Unauthorized use, reproduction, or disclosure is prohibited. Patents pending. No patent liability is assumed with respect to the use of the information contained herein. While every precaution has been taken in the preparation of this user's guide, ARANZ Medical assumes no responsibility for errors or omissions. Neither is any liability assumed for damages resulting from use of the information contained herein.





ARANZ Medical Limited
Tel +64-3-374 6120, Fax +64-3-374 6130

Postal Address:

ARANZ Medical

PO Box 3894

Christchurch 8140

New Zealand

Delivery Address:

ARANZ Medical

47 Hereford Street

Christchurch 8013

New Zealand

**CE Authorised Representative** 

mdiEUROPA

Phone +49 (0)511 39089530

werner.sander@mdi-europa.com

Langenhagener Str. 71 30855 Langenhagen



FDA Authorised Representative TGA Sponsor #55050

MDI Consultants Brandwood Biomedical Pty Ltd Phone +1 (516) 482 9001 Phone +61 (0) 2 9906 2984

alan@mdiconsultants.com help@brandwoodbiomedical.com

ARANZ Medical Limited documentation often refers to hardware or software products by their trade names. In most, if not all, cases these designations are claimed as trademarks or registered trademarks of their respective companies. The trade names are used here for identification purposes only.

Document Number: 2018-00156 Rev2.0

# **Contents**

| Introduction to Silhouette                                            | 1  |
|-----------------------------------------------------------------------|----|
| Document Conventions                                                  | 1  |
| System Description                                                    | 1  |
| Topics                                                                | 3  |
| Using Silhouette                                                      | 4  |
| Logging In to Silhouette                                              | 5  |
| Changing Passwords                                                    | 5  |
| Recovering Forgotten Password                                         | 6  |
| Navigating in Silhouette                                              | 8  |
| Application Menu                                                      | 8  |
| Identifying Patients                                                  | 9  |
| Searching for Patients                                                | 9  |
| Patient Sidebar                                                       | 10 |
| Capturing Wound Images                                                | 11 |
| Connecting a SilhouetteStar camera to SilhouetteConnect using USB     | 11 |
| Connecting a SilhouetteStar 2 camera to SilhouetteCentral using Wi-Fi | 12 |
| Using the SilhouetteStar camera to capture images                     | 12 |
| Disconnecting the SilhouetteStar camera                               | 13 |
| Logging Out from Silhouette                                           | 14 |
| Silhouette Administration                                             | 15 |
| Groups, Users, and Units in Silhouette                                | 16 |
| Automated Groups, Users and Units Set Up                              | 17 |
| Users                                                                 | 18 |
| Creating a User                                                       | 18 |
| Editing a User                                                        | 18 |
| Groups                                                                | 20 |
| Creating a Group                                                      | 20 |
| Group Details                                                         | 20 |
| Group and Unit Relationship                                           | 21 |
| Group Permissions                                                     | 21 |
| Editing a Group                                                       | 24 |
| Assessments                                                           | 25 |
| Editing an Assessment Type                                            | 26 |
| Patient Notes                                                         | 28 |
| Editing Patient Notes                                                 | 28 |
| Unit Administration                                                   | 29 |
| Editing a Unit                                                        | 30 |

| SilhouetteStar 2                                              | 31 |
|---------------------------------------------------------------|----|
| SilhouetteStar 2 DHCP Configuration                           | 31 |
| Checking the SilhouetteStar 2 DHCP Configuration              | 32 |
| Network Configurations and Wireless Camera Tools              | 33 |
| Creating a Network Configuration                              | 34 |
| Editing a Network Configuration                               | 35 |
| System Requirements and Operating Conditions                  | 36 |
| SilhouetteCentral Web Application - Minimum Requirements      | 36 |
| SilhouetteConnect - Minimum Requirements                      | 36 |
| SilhouetteStar 2 - Minimum Requirements                       | 36 |
| SilhouetteLite and SilhouetteLite+ App - Minimum Requirements | 37 |
| SilhouetteLite+ Sensor - Operating Conditions                 | 38 |
| Appendices                                                    | 39 |
| Appendix A: Getting Support                                   | 40 |
| Appendix B: Anatomical Sites                                  | 41 |
| Head & Neck                                                   | 41 |
| Torso                                                         | 43 |
| Pelvis                                                        | 44 |
| Upper Extremity                                               | 45 |
| Lower Extremity                                               | 48 |
| Appendix C: Notes Definitions                                 | 50 |
| Patient admission                                             | 50 |
| Details                                                       | 50 |
| Physicians                                                    | 50 |
| Patient Details                                               | 51 |
| Details                                                       | 51 |
| Address                                                       | 52 |
| Contact                                                       | 52 |
| Patient discharge                                             | 53 |
| Details                                                       | 53 |
| Assessment of lower limb function                             | 53 |
| Right                                                         | 53 |
| Ambulation                                                    | 56 |
| Left                                                          | 56 |
| Comments                                                      | 58 |
| Visit Assessment                                              | 59 |
| Vital signs                                                   | 59 |
| Related Pain (Non-Wound)                                      | 60 |
| Edema                                                         | 60 |

| Factors Affecting Healing                  | 61 |
|--------------------------------------------|----|
| Wound Assessment                           | 63 |
| Images                                     | 63 |
| Details                                    | 64 |
| Wound Pain                                 | 69 |
| Comments                                   | 71 |
| Investigation History                      | 71 |
| Investigation History                      | 71 |
| Medical History                            | 72 |
| Medical History                            | 72 |
| Medication                                 | 74 |
| Medication                                 | 74 |
| Sensitivities                              | 74 |
| Appendix D: Regular Expressions            | 76 |
| Patient Identifier Examples for Silhouette | 76 |
| Example A                                  | 76 |
| Example B                                  | 76 |
| Example C                                  | 76 |
| Example D                                  | 76 |
| Other examples                             | 77 |
| Things to Note                             | 77 |

## **Introduction to Silhouette**

This guide is intended for Silhouette administrators and is focused on the system administration features. Other user guides available:

- The Silhouette Installation and Configuration guide, which covers the installation and initial configuration of SilhouetteCentral and SilhouetteConnect.
- The Silhouette Clinical User's Guide, which covers the clinical features for clinical users.

## **Document Conventions**

Throughout this user's guide, screen, menu, and field names in Silhouette are displayed in a **bold font**. Series of actions are referred to as follows:

## Admin > Organization > Settings

which means to select the Settings option in the Organization menu in the Silhouette Admin section.

The screens your browser displays may differ slightly from the screen captures shown in this document. However, such differences are minor and do not affect the performance of Silhouette.

Throughout this user's guide, references to Silhouette mean both the SilhouetteConnect Microsoft Windows application and the SilhouetteCentral web application. Where a feature or function is only applicable to one of the Silhouette applications then that application is specifically stated.

## System Description

SilhouetteCentral is a computerized wound imaging, analysis, and documentation system. SilhouetteCentral enables users to review, report, securely share and analyze the wound data collected by SilhouetteConnect, SilhouetteLite+ and SilhouetteLite applications. SilhouetteCentral is accessed via a web-based application and can collect wound data directly with the SilhouetteStar 2 camera.

SilhouetteCentral is divided into two parts:

- Clinical UI Provides access to the patient medical data stored in the Silhouette database for reviewing, editing, and updating. Reports, images, and wound measurement data can also be exported for further processing and importing into other applications.
- Administration Provides facilities to manage units, groups, patients, and system users. SilhouetteCentral can also be used to configure and customize the Silhouette system.



[2] In this version of SilhouetteCentral there is limited configurability available via the main web app. Contact ARANZ support if configuration changes need to be made.

SilhouetteConnect is the software installed on a Microsoft Windows tablet, laptop or desktop computer. SilhouetteStar cameras connect via USB to a computer with SilhouetteConnect. SilhouetteConnect enables wound assessments to be performed without connection to a network, and the data is synchronized with SilhouetteCentral when a network connection is available.

#### 2 • System Description

SilhouetteConnect can be used in two modes of operation:

- Standalone mode provides the power of Silhouette on a single PC. Data is collected and stored on the PC. PDF reports can be generated to enable sharing of the results.
- Synchronized mode provides offline and wired camera support for SilhouetteCentral systems. Data collected by SilhouetteConnect can be uploaded to SilhouetteCentral and data stored in SilhouetteCentral can be downloaded and used offline in SilhouetteConnect.

In synchronized mode the majority of the administration for the system is controlled by SilhouetteCentral.



[2] In this version of SilhouetteConnect there is limited configurability available via the main web app. Contact ARANZ support if configuration changes need to be made.

SilhouetteStar 2 is a camera that captures wound images and non-contact 3D measurements of wounds at the point of care. The images and data captured with SilhouetteStar 2 are uploaded to SilhouetteCentral over a secure Wi-Fi network or to SilhouetteConnect over a USB cable.

SilhouetteStar 1 is a camera that captures wound images and non-contact 3D measurements of wounds at the point of care. The SilhouetteStar 1 operates over a USB cable connected to a computer running SilhouetteConnect software.

SilhouetteLite+ is comprised of an application and a range finding sensor for Apple® iPhone®, iPod® touch and iPad® devices. SilhouetteLite+ enables users to take wound images, obtain non-contact 2D measurements, and record patient notes on their mobile device. The data collected at the patient's bedside is synchronized with SilhouetteCentral over a secure connection when available.

SilhouetteLite is an application designed for Apple® iPhone®, iPod® touch and iPad® devices. SilhouetteLite is a simplified application that enables users to add patients, wound images and simple notes (without measurements) to SilhouetteCentral. The data collected at the patient's bedside is synchronized with SilhouetteCentral over a secure network connection when available.

# **Topics**

This user's guide is divided into the following high level topics:



**Using Silhouette** 



**System Requirements and Operating Conditions** 



Silhouette Administration



**Appendices** 

# **Using Silhouette**

This section describes the basics of using with Silhouette. It is divided into the following sections:







Navigating in Silhouette



Logging Out from Silhouette

## Logging In to Silhouette

To log in to SilhouetteCentral, navigate to the website address of your SilhouetteCentral instance using a Web Browser.

To log in to SilhouetteConnect, start the SilhouetteConnect application on your computer.

The log on screen will appear. Enter the User Name and Password supplied to you. Select Login.



It is important to log on using the **User Name** and **Password** assigned to you. Silhouette records the names of users viewing, creating, and modifying patient records, and using the correct user name ensures accurate auditing.

While logging on to Silhouette you may occasionally be prompted to change your password if a system administrator has made use of the password expiry features. In this case you will need to pick a new password that meets the password complexity requirements and enter it before Silhouette will allow you to log on.

When logging in to SilhouetteCentral, web browsers may prompt you to save a **User Name** and **Password**. If using a shared computer, it is recommended that you do not permit the browser to save your login detail for SilhouetteCentral. Doing so, means that anyone using your computer can log on, access and potentially modify patient medical records that will be recorded as having been completed by you.

## **Changing Passwords**

For users with permission to manage Users and Groups, your own and others passwords are managed via the <u>Users</u> screen

Note that user names and passwords are used across the Silhouette Product Suite, so you will also need to use the new password when logging in to SilhouetteCentral, SilhouetteConnect, SilhouetteLite or SilhouetteLite+.

For users without permissions to manage Users and Groups to change their Silhouette password:

- 1. Navigate to the Admin > Profile screen.
- 2. Select the Change Password button.
- **3.** Enter your existing password in the **Current Password** box.
- 4. Enter your new password in both the New Password and Confirm Password boxes.
- 5. Select the Save button to update the system with your new password.

When SilhouetteConnect is operating in Synchronized mode, in addition to changing their password in SilhouetteCentral, as above, all users can change their password by accessing **Admin > Profile** in SilhouetteConnect. A synchronization must be performed before the password change is applied across the system. During synchronization SilhouetteConnect needs to ask for the user's old Silhouette password (which SilhouetteCentral still has) to successfully complete. If the old password can't be remembered the password can be reset via SilhouetteCentral, before the sync can proceed.

#### 6 • Recovering Forgotten Password

Logging in to an offline SilhouetteConnect after a password has been changed in SilhouetteCentral requires the users original password until a synchronization is completed. When SilhouetteConnect is online, it allows the new password to be used.

## **Recovering Forgotten Password**


If you have forgotten your password, use the "Forgot your Password?" link on the SilhouetteCentral log in screen. This link is only available in SilhouetteCentral, not SilhouetteConnect. If you do not see this link, your administrator may have chosen to disable this feature, in which case contact your system administrator for further advice on how to regain access to your account.




Once the "Forgot your password?" link is selected you will be prompted to enter your email address. You should enter the email address that is associated with your Silhouette user name.



You will get a message that an email with instructions to reset your password has been sent to the email address you specified.

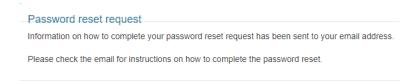


When you check your email account and receive the email sent by Silhouette you will find a link that can be used to reset your password. Selecting the link will load the Silhouette website and prompt you to enter a new password.

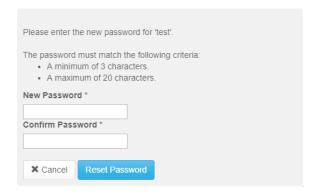

You will receive a confirmation that your password has been changed and you can now proceed to log on with the new password.



If you receive a password reset request email and do not remember requesting it, an additional link in the email may (depending upon system configuration) be provided that allows you to cancel the password reset request so any other person who manages to view the email can not reset your password.

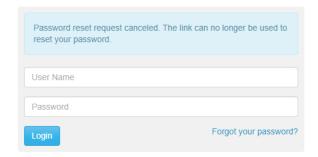

In some circumstances you may get a password reset email that indicates your user account is not able to have its password reset via email. In this scenario contact your system administrator for further instructions on how to reset your password.


## **Navigating in Silhouette**

The basics of navigating within Silhouette are described in the following topics:

- Application Menu
- Identifying Patients
- · Searching for Patients
- Patient Sidebar
- Capturing Wound Images

## **Application Menu**

Once you have logged in to Silhouette, your name is shown in the top right-hand corner, with a drop down menu, known as the Application Menu:

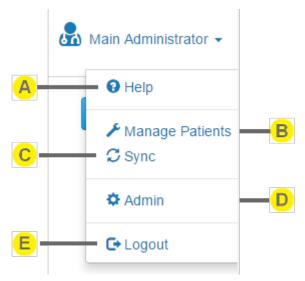



## Help

Contains links to the User Guides to help you use Silhouette. It also contains the links to legal and licensing information for Silhouette.



## **Manage Patients**

Opens the Manage Patients screen on SilhouetteConnect, allowing the selection of patients to download to this instance of SilhouetteConnect.

This menu item is only shown in SilhouetteConnect if it is running in Synchronized mode and if the current user has the **Can Manage Patients** permission.



## Sync

Initiates a Sync to SilhouetteCentral.

This menu item is only shown in SilhouetteConnect if it is running in Synchronized mode and if the current user has the **Can Synchronize** permission.



#### **Admin**

Change your account settings such as your name, email address and password.

Also, if your account has the correct permissions, then you can administer settings for the application. <u>See</u> "Silhouette Administration".



## Logout

Log out of SilhouetteCentral. See "Logging Out from Silhouette".

## **Identifying Patients**

Silhouette uses three (3) pieces of information to positively identify patients. By default, this information is setup for clinical practice environments:

- Patient name (first name + last name)
- Patient ID
- · Patient Date of Birth

These bits of information are displayed throughout the system to help positive identification of the patient being imaged.

Silhouette requires that every patient in the system has an ID. The patients name and date of birth can be made optional for clinical research focused deployments.

Contact ARANZ Medical support to make changes to the patient identifiers.

## **Searching for Patients**

In some Silhouette screens, you can search for patients by their first name, middle name, last name, or Patient ID.





#### Search box

Enter part of the patient's name or Patient ID. The search will be performed after the typing has stopped.

For example, searching for the letters "an" filters the list so that it displays patients whose name or Patient ID begins with the letter sequence "an".

Narrow the search further by entering more of a patient's name or Patient ID.




## **Clear button**

Select the **x** button to clear the search box and display the full list of patients.

## **Patient Sidebar**

Once a patient is selected a sidebar is displayed that can be used for navigating between sections. The current section is highlighted in blue.

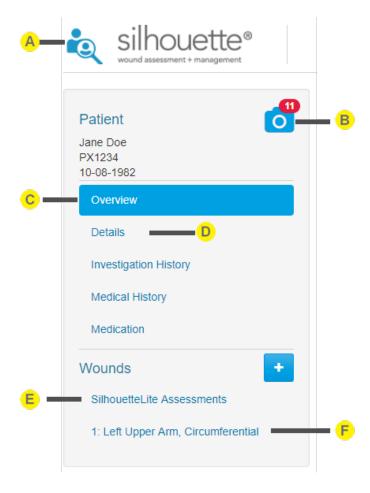

- Select the patient search icon or the logo to go back to the Patient Select screen
- The camera icon opens the **Portfolio** dialog, containing the unassigned captured images and the SilhouetteStar connection information.
- Opens the Patient Overview



Opens the Patient Details (name, date of birth, etc). The patient note sections available on the sidebar are configurable in the **Admin** section.



View all the SilhouetteLite assessments



Opens the **Wound Overview** for the selected wound

Note that it is possible to use multiple tabs to view SilhouetteCentral. Care should be taken to ensure the correct patient data is being reviewed when using this feature.

## **Capturing Wound Images**



7 To get more comprehensive help with the Clinical UI in Silhouette, please see the Clinical User's Guide.

In order to capture wound images, Silhouette must be connected to a SilhouetteStar camera.

## Connecting a SilhouetteStar camera to SilhouetteConnect using USB

SilhouetteConnect allows connection of either the SilhouetteStar 1 or the SilhouetteStar 2 camera via a USB cable.

The USB connected camera automatically connects, if plugged in, once as the user is logged in to SilhouetteConnect.

To check that the camera is detected and connected, either check the screen of the SilhouetteStar 2 camera or navigate to the portfolio dialog for a patient. The Portfolio dialog is accessed by selecting the camera icon on the Patient sidebar.



The portfolio dialog provides a message indicating that the camera is connected and is ready to capture images.

## Connecting a SilhouetteStar 2 camera to SilhouetteCentral using Wi-Fi

SilhouetteCentral allows connection of the SilhouetteStar 2 camera using Wi-Fi. For this, Silhouette needs at least one SilhouetteStar 2 network configured. See <u>Creating a Network Configuration</u>.

Connect the camera to SilhouetteCentral via Wi-Fi by opening the Portfolio dialog. To do this, select the **camera icon** on the Patient sidebar.



If the Portfolio shows the captured images instead of the networks available, use the 'Connect the SilhouetteStar' link.

The dialog will display a QR code if there is only one network available. If there are multiple networks configured, it will show a dropdown so that the correct network can be selected.

Press the button on the SilhouetteStar 2 camera to turn on the lasers. Hold the camera with the button facing up and point the lasers at the QR code to connect to Silhouette.

Once it has connected, the Patient details will be shown on the SilhouetteStar 2 camera display, and images can be captured.

## Using the SilhouetteStar camera to capture images

Images can be captured whenever the SilhouetteStar camera (SilhouetteStar 1 or SilhouetteStar 2) is connected and there is a patient selected.

To capture images:

- Press the button on the SilhouetteStar camera to turn the lasers on.
- Form a star with the lasers inside the wound, making sure one laser is crossing the deepest part of the wound.
- Press the button on the SilhouetteStar camera to take the image, holding the camera as still as possible for approx. 2 seconds. The SilhouetteStar 1 camera initiates the image capture when the button is pressed, while the SilhouetteStar 2 camera initiates the image capture when the button is released.
- The lasers turn off after a timeout. If the lasers turn off and there are still more images to be captured, then press the button to turn them on again.

Captured images are uploaded to the selected patient's portfolio and can be used when creating assessments. If using SilhouetteConnect, the images in the patient's portfolio are stored locally on the computer and are not synchronized to SilhouetteCentral. Images stay in the portfolio until they are used in an assessment or until the are deleted by the user.

The user is able to navigate between patients in Silhouette while the camera is connected. When using a Wi-Fi camera connection to SilhouetteCentral then the camera connection is linked to the browser tab to which it was originally connected. If that browser tab is closed then the camera is disconnected and must be re-connected to another browser tab.

## **Disconnecting the SilhouetteStar camera**

The SilhouetteStar camera will remain connected until the Silhouette session ends (e.g. by logging out, by application lock timeout, by closing the web browser, etc).

When operating in Wi-Fi mode, the SilhouetteStar 2 camera will go to sleep after a period of inactivity to preserve the battery power. When the button is pressed, the camera will wake up and attempt to reconnect to the last session if it is still open.

If using the SilhouetteStar 2 camera the battery in the camera still needs to be changed, even when operating in a USB connected configuration.

# **Logging Out from Silhouette**

You can log out of Silhouette by using the Application Menu in the upper right-hand corner, and selecting the **Logout** option.

Note that you will be automatically logged out of Silhouette after a period of inactivity. This period is configurable, but by default is 10 minutes.

# **Silhouette Administration**

Silhouette administration is controlled through either SilhouetteCentral or SilhouetteConnect, see <u>Using Silhouette</u>.

The administration options available depend on what application you are using and what mode it is operating in. Each administration topic identifies specific application and mode dependency. A high level description of application and mode dependency is listed here:

- SilhouetteCentral (any mode) contains most administration, except wired mode specific settings for SilhouetteStar 2.
- SilhouetteConnect, Synchronized mode contains only administration settings applicable to the administration of the specific PC. All system wide settings are configured in SilhouetteCentral and updated in SilhouetteConnect when a synchronization is performed.
- SilhouetteConnect, Standalone mode contains a subset of all system administration. Settings that are not applicable to standalone (non-enterprise) settings are not available.

The admin section is accessed through the **Application Menu** and is organized into sections, which are displayed in the sidebar.









Users

**Groups** 

<u>Assessments</u>




8

The sections displayed to any given user will depend upon the permissions that have been configured for that users assigned group.

## Groups, Users, and Units in Silhouette

Access to the Silhouette data is managed through units and groups. Additionally, the group controls functionality available to the users.

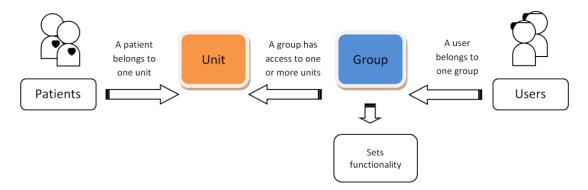

## Where:

- Every patient belongs to a single unit. While a patient can only belong to one unit at a time, this can change over time
- Each user belongs to a single group.
- A group can be associated with one, several, or all units of the organization.

## A Silhouette user has access to:

- patients that belong to the unit(s) to which the user's group is associated with; and
- perform actions that are allowed by the user's group permissions.

As an example, your organization might have a set of users who belong to the same group that can only access patients from a specific unit in the organization. A user might also be limited in what changes they can make to patient records or might be prevented from creating new patients in the system. At the same time, there may be a set of users who belong to a different group that has access to every patient and can use any Silhouette feature available in the system.

Organizations use units in different ways, depending upon how the organization is structured. For example, an organization may use units to differentiate different departments, wards, or facilities. Clinical research organizations typically create a unit per research site.

Putting patients into Units within Silhouette is an important way of controlling who has access to patient data. It also controls how much patient data can be stored in offline devices using SilhouetteConnect, SilhouetteLite+ or SilhouetteLite.

Units can be defined in the <u>Unit Administration</u> screen. Group settings, including which units members of a group can access, are defined in the <u>Groups</u> screen. Users can be created and edited in the <u>Users</u> screen, which allows assignment of a user to one of the groups defined in the <u>Groups</u> screen.

When configuring Silhouette for your organization, first define and configure Units, then Groups and finally users.

## Automated Groups, Users and Units Set Up

If you have access to the physical server running SilhouetteCentral you can alternatively make use of a more direct import process, as an alternative to using the administration user interface to configure additional groups, users, or units.

Files with the following special filenames can be placed in the Files\Setup folder of SilhouetteCentral's installation folder and will be automatically detected and processed during the next application pool restart. If successfully processed, the file will be deleted from the Files\Setup folder to indicate its successful import.

- Units.csv a comma separated list of units to import into Silhouette.
- Groups.csv a comma separated list of groups to import into Silhouette.
- Users.csv a comma separated list of users to import into Silhouette.


As an example if you wanted to quickly add 50 additional units to a Silhouette installation you could create a Units.csv file describing the details of the 50 additional units and then place this file in the Files\Setup folder. Once the file is in place, restarting the IIS application pool would cause the 50 units to be imported into the system.

Please contract ARANZ Medical Limited support for assistance preparing the csv files for importing.

## **Users**

Users must have a user account to access any of the Silhouette applications (SilhouetteCentral, SilhouetteLite, or SilhouetteLite+). The **Users** screen allows you to add and manage Silhouette user accounts.

To define and manage users, select **Admin > Users**. The **Users** option is not available in SilhouetteConnect when it is running in synchronized mode. In Synchronized mode, user administration is performed in SilhouetteCentral.

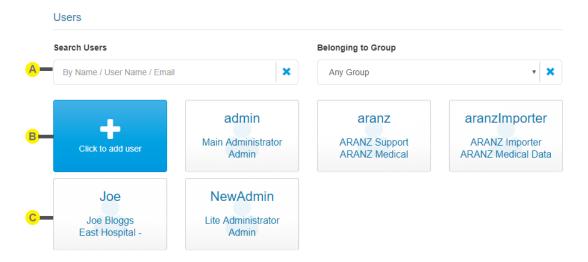



Limit the users shown in the list using these options.

B Create button
See "Creating a User".

User tile
Select the user tile to view or edit the user. See "Editing a User".

## Creating a User

To add a new user to Silhouette, in the **Users** screen select the **Click to add user** button. The **Create User** screen is displayed, with compulsory fields marked with an asterisk. Enter the required information.


Note that every user within the organization must have a unique **User Name**.

The **Group** a user is assigned to determines what Silhouette features and patients the user has access to. See <u>Groups</u> for more information.

When you have entered the user's details, select the **Save** button at the bottom of the screen to save the user's information. After saving you are returned to the **Users** screen.

## **Editing a User**

To edit an existing user's details (including your own), select that user's tile in the **Users** screen. The information for the user is displayed in a read-only view.

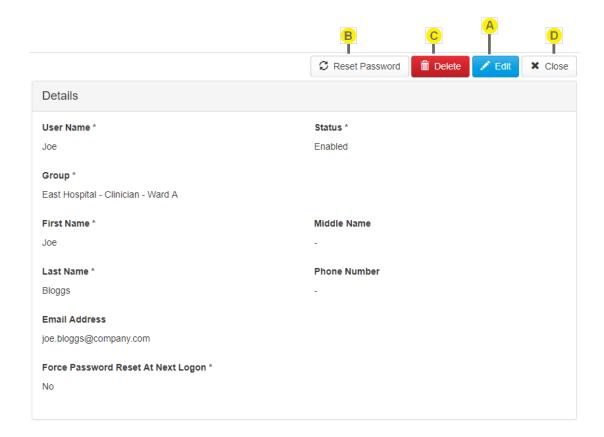



## Edit

Select the **Edit** button to change from read-only mode to edit mode.

In Edit mode, compulsory fields are marked with an asterisk. After editing the user's details, select the **Save** button at the bottom of the screen to save the user's information and return you to the read-only view.



## **Reset Password**

Select **Reset Password** to change the user's password.



## Delete

Select **Delete** to delete a user. A confirmation prompt is presented before the user is deleted.

A user can be disabled instead of deleted by selecting **Edit** and changing the **Status** field. The **Status** drop-down controls if a user can log in to Silhouette. A user is able to log in when their status is "Enabled". To temporarily stop a user from logging in, you can set their status to "Disabled". The status is set to "Locked" if the system has automatically disabled an account. The distinction between "Disabled" and "Locked" can be used to determine which user accounts have been explicitly disabled by a system administrator, and those which a user has accidentally disabled due to entering the incorrect password too many times.



## Close

Select **Close** to go back to the <u>Users</u> screen.

## **Groups**

To ease support burdens, individual user accounts are assigned to groups based upon similar needs with respect to the features and patients they are allowed to access. This allows the permissions to be configured once and applied immediately across a number of user accounts.

As an example, you may create a group to allow data correction tools to be utilized by research staff, while another group containing your clinical staff would not have access to these tools. Groups can also be used to restrict users to interacting with a subset of patients (via units).

To define and manage groups, select **Admin > Groups**. The **Groups** option is not available in SilhouetteConnect when it is running in synchronized mode. In Synchronized mode, group administration is performed in SilhouetteCentral.







## Group tile

Select the group tile to view or edit the group. See "Editing a Group".

## **Creating a Group**

To create a new group, in the **Group Administration** screen select the **Click to add group** button. The **Create Group** screen is displayed, with compulsory fields marked with an asterisk.

The Save and Cancel buttons are at the bottom of the screen.

Select the **Save** button at the bottom of the screen to create the group.

Select the **Cancel** button at the bottom of the screen to exit the screen without creating the group.


## **Group Details**

Enter a Group Name and Description.



## **Group and Unit Relationship**

Each group must select which Units that the group permissions apply to in the **For The Following Units** section. The selection of unit limits the patients, users and groups that the group members can access. <u>See "Groups, Users, and Units in Silhouette"</u> for more information on how the relationship between Users, Patients, Groups and Units work.

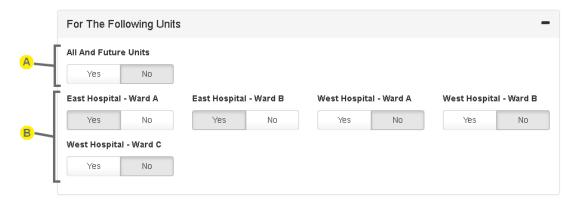



## **All And Future Units**

Select Yes for **All And Future Units** to give the group access to all existing units and all new units as they are created within Silhouette.

Select No for All And Future Units to manually select which individual units the group has access to.



## Individual unit selection

Select which units this group of users has access to.

Selecting Yes to All And Future Units hides the individual unit selection options.

When a new unit is created in the system then any existing groups do not automatically get permission to access it.

## **Group Permissions**


Below is a description of each permission that can be assigned to a group. The permissions are divided up into areas:

- Clinical Data Permissions.
- · Silhouette Permissions.
- · Administrator Permissions.

The permissions listed in italics below are intended to control features that are not yet implemented in the current version of Silhouette and so do nothing as yet.

## **Clinical Data Permissions**

Clinical Data Permissions control the access to patient data.

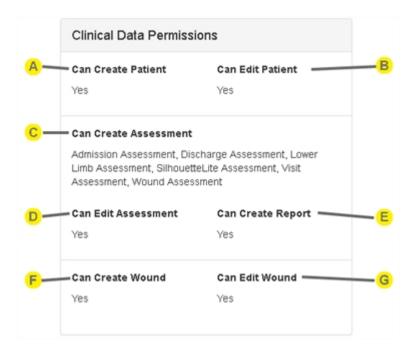

## Can Create Patient

Controls if users assigned to this group can create patients.

## Can Edit Patient

Controls if users assigned to this group can edit patient details or any of the patient notes.

The ability to view the patient notes in the Clinical UI is also linked to the application the user is using. <u>See</u> "Patient Notes" to show how to control which patient notes are visible in each of the Silhouette applications.

## Can Create Assessment

Controls which assessments (if any) the user is allowed to create. Creating assessments includes the ability to use images from the portfolio in an assessment (assigns the image to a wound) and perform measurements with that image.

This is a multi-selection list box, listing all of the assessment types available in the system. Leaving the selection list empty means that the user can not create any assessments.

The ability to create different assessments is also linked to the application the user is using. <a href="See">See</a> <a href=""Massessments">"Assessments"</a> to show how to control the assessments that are creatable in each of the Silhouette applications.

# Can Edit Assessment

Controls if users assigned to this group can edit assessments. Editing assessments includes the ability to add or remove images and edit image tracings after the initial assessment creation.

# Can Create Report

Controls if users assigned to this group can create reports for assessments.

# Can Create Wound

Controls if users assigned to this group can create wounds.



#### **Can Edit Wound**


Controls if users assigned to this group can edit wounds.



A group with permissions to a unit can view all patients, wounds and assessments for that unit, i.e. there is no separate view permission above.

#### Silhouette Permissions

Silhouette Permissions control the access to system functions.

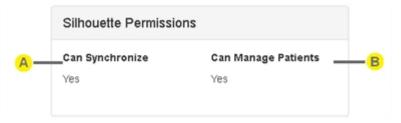



## **Can Synchronize**

Controls if users assigned to this group can synchronize a SilhouetteConnect with SilhouetteCentral.




## **Can Manage Patients**

Controls if users assigned to this group can add/remove patients from the offline dataset held in a SilhouetteConnect install. Groups with Can Manage Patients also need Can Synchronize permission.

#### **Administrator Permissions**

Administrator permissions control access to various sections in the admin sidebar.

Most of the options listed under administrator permissions are only available if the group is set to access All And Future Units. The administration permissions that are not available are hidden if All And Future Units is set to No. This is because some administrator functionality allows interaction with any patient within the system, or affects all users. See the description of Can Manage Users and Groups below.

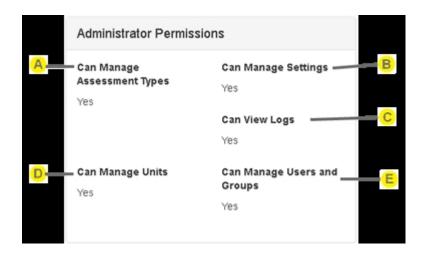



## **Can Manage Assessment Types**

Controls if users assigned to this group can edit assessment types.

This permission controls the visibility of the **Admin > Assessments** and the **Admin > Patient Notes** items in the Silhouette admin sidebar.



## **Can Manage Settings**

Controls if users assigned to this group can access the organization settings.

This permission controls the visibility of the **Admin > SilhouetteStar 2** item in the Silhouette admin sidebar.



## Can View Logs

Controls if users assigned to this group can view the logs.



#### **Can Manage Units**

Controls if users assigned to this group can add/edit/delete units.

This permission controls the visibility of the Admin > Units item in the Silhouette admin sidebar.



## **Can Manage Users and Groups**

Controls if users assigned to this group can add/edit/delete users and groups.

This permission controls the visibility of the **Admin > Users** and **Admin > Groups** items in the Silhouette admin sidebar.


If a group does not have this permission then the users will have access to **Admin > Profile** to see their own user profile and will be able to change their own password.

The **Can Manage Users and Groups** administrator permission remains available, even when the group does not have access to **All And Future Units**. The scope of this permission is set by the Unit selection for the current group, allowing for the creation of limited administrators.

A limited administrator is a user who can manage users and groups for a limited set of units. A limited administrator is created by assigning a user to a group that has access to a limited set of units and has **Can Manage Users and Groups** enabled. A limited administrator can manage users and groups that have access to the same set (or a subset) of units that the limited administrator has access to. When creating or editing a group, a limited administrator can only select unit(s) for the group that the limited administrator has access to.

## **Editing a Group**

To edit a group to Silhouette, in the **Group Administration** screen, select the group from the groups list. The information for the group is displayed in a read-only view.

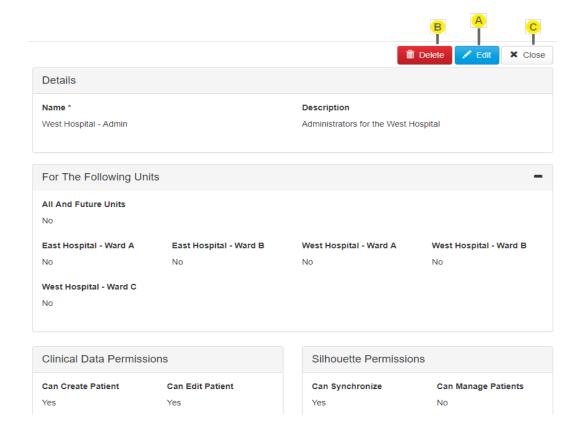



## Edit

Select the **Edit** button to change to edit mode from read only mode. Update the information, as required. To save the group's settings, select the **Save** button at the bottom of the screen.

A user cannot edit the **Units** and **Can Manage Users and Groups** permission for its own group in order to prevent them from accidentally changing permissions they cannot restore. For example, if user has access to all units and was to change their own group permissions to limited unit access, they would then not be able restore their own permissions to all units as the system would see this as an attempt to escalate their own access to patient data. However, the user change the **Can Create Patient** permission for their own group, as they can always restore that permission.



#### Delete

Select the Delete button to delete the group. Before deleting you will be asked to confirm the delete operation.

If a group has users assigned to it, then it can not be deleted.



## Close

Select the Close button to go back to the **Groups** screen.

#### **Assessments**

Assessments are the main charting mechanism within Silhouette. Assessments appear on a patient's timeline and represent a set of data recorded at a point in time. Silhouette allows for two different types of assessments:

• Patient Assessments, which are about the patient as a whole (rather than a particular wound) and are useful to record patient risk assessments or similar whole of patient assessments.


#### 26 • Editing an Assessment Type

• Wound Assessments, which are about a particular wound on the patient and are the main assessment used to track progress of a given wound.

Silhouette comes with some pre-loaded Patient Assessments and Wound Assessments. Extra assessments can be added to the system to suit your organization.

Assessments are different to patient notes, which appear in the patient sidebar and do not relate to a point in time. For example, a patients allergies may change over time but they are not assessed every week. See "Patient Notes" for help administering patient notes.

To define and manage assessments, select **Admin > Assessments**. The **Assessments** option is not available in SilhouetteConnect when it is running in synchronized mode. In Synchronized mode, assessment administration is performed in SilhouetteCentral.

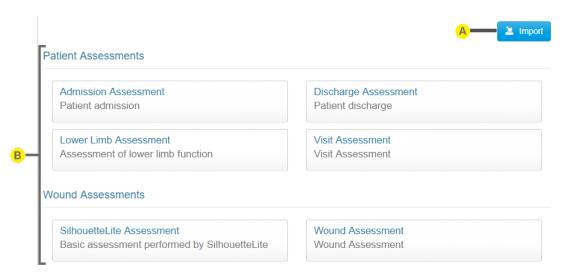



#### Import button

Allows an assessment definition zip file to be imported, which either creates a new assessment or modifies the definition of an existing assessment.

A newly imported Assessment is not be accessible to any users by default. See "Editing a Group" for information on granting users access to create the new Assessment.

Contact ARANZ Medical Limited for help to configure the assessments you need.




#### **Assessment Tiles**

Selecting an assessment opens the assessment definition to show the detail and allow editing, <u>See</u> "Editing an Assessment Type".

Use caution when modifying the definition of existing assessments when there are users using your system offline with either SilhouetteConnect or SilhouetteLite(+). Simple changes, such as making a field required, may cause sync errors when offline data created with previous definitions is attempting to sync.

## **Editing an Assessment Type**

To edit an assessment in Silhouette, in the **Admin > Assessments** screen, select the assessment from the list. The information for the assessment is displayed in a read-only view.

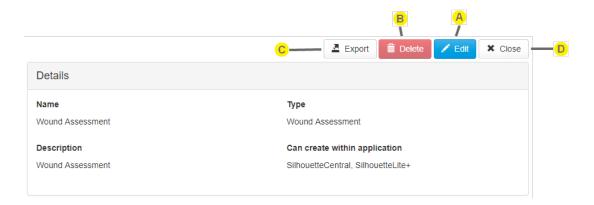



## Edit

Select the **Edit** button and update the information as required.

- The **Name** is displayed to users when they create assessments and is displayed at the top of the assessment.
- The **Type** can be Wound Assessment or Patient Assessment.
- The **Description** is provided to show as additional detail to the user when they create the assessment.
- The Can create within application allows the selection of which application can be used to create an
  assessment of this type. Selecting SilhouetteCentral means both SilhouetteCentral and
  SilhouetteConnect.

Once edited, select the **Save** button at the bottom of the screen to save the settings.



## Delete

Select the **Delete** button to delete the assessment type. The system prevents you from deleting assessment types which have been used to create assessments for patients. If you wish to prevent more assessments being created of a given type then disable new assessments by unticking all applications under the **Can create within application** option.



## **Export**

Select the **Export** button to download a zip file containing the assessment type definition. The assessment type can then be imported into another Silhouette system (e.g. between a test and production system) or provided to ARANZ Medical for modification.




#### Close

Select Close to go back to the Assessments screen.

## **Patient Notes**

The Patient Notes admin section shows the patient notes categories which appear on the patient sidebar (includes Details, Investigation History, Medical History and Medication by default). Patients notes can be added or removed from the system to suit your organization.

To define and manage Patient Notes, select **Admin > Patient Notes**. The **Patient Notes** option is not available in SilhouetteConnect when it is running in synchronized mode. In Synchronized mode, assessment administration is performed in SilhouetteCentral.

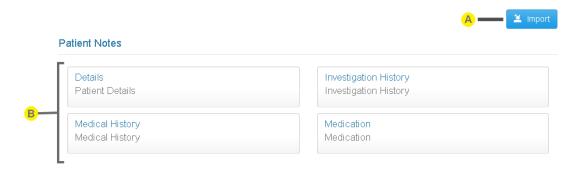



## Import button

Allows patient notes definition zip file to be imported, creating a new patient note or modifying an existing patient notes.

Contact ARANZ Medical Limited for help to configure the Patient Notes you need.




#### **Patient Note tile**

Select the patient note to view and edit the details. See "Editing Patient Notes"

## **Editing Patient Notes**

To edit Patient Notes setup in Silhouette, select the patient note tile from the list of patient notes in the **Admin > Patient Notes**. The information for the patients notes is displayed in a read-only view.

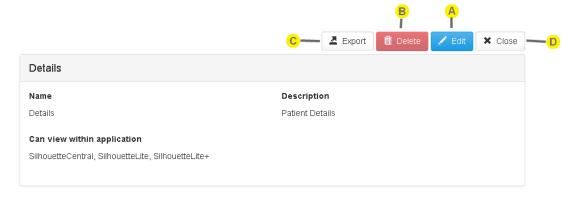



#### Edit

Select the **Edit** button to update the information, as required.

- The Name is displayed in the patient sidebar.
- The **Description** allows some text to help keep track of what the patient notes are for.
- The **Can view within application** controls where the patient notes are visible. SilhouetteCentral means both SilhouetteCentral and SilhouetteConnect. Selecting SilhouetteLite or SilhouetteLite+ has no effect as these applications do not have features to view the patient notes.

Once edited, select the **Save** or **Cancel** button at the bottom of the screen to save or cancel the changes.



#### **Delete**

Select the **Delete** button to delete the patient notes from the system. The system allows you to delete patient notes even if there are some patients with notes recorded against them.



## **Export**

Select the **Export** button to download a zip file containing the patient notes definition. The patient notes can then be imported into another Silhouette system (e.g. between a test and production system) or provided to ARANZ Medical for modification.



## Close

Select **Close** to go back to the **Patient Notes** screen.

## **Unit Administration**

A unit defines an entity which patients can be assigned to. Different organizations use units in different ways, depending upon how the organization is structured. For example, an organization may use units to differentiate different departments, wards, or facilities.

To define and manage units, select **Admin > Units**. The **Units** option is not available in SilhouetteConnect when it is running in synchronized mode. In Synchronized mode, assessment administration is performed in SilhouetteCentral.





## **Create button**



## **Unit tile**

Select the tile for a unit to view and edit the unit, See "Editing a Unit".

## **Editing a Unit**

To edit a unit in Silhouette, in the **Unit Administration** screen, select the unit from the units list. The information for the unit is displayed in a read-only view.





## Edit

Select the Edit button to change from ready-only view to editing mode.

• The **name** is used through-out the system to represent the Unit, including when creating and editing patients.

Once edited, select the **Save** or **Cancel** button at the bottom of the screen to save or cancel the changes.



## Delete

To delete the unit, use the **Delete** button.

Silhouette does not allow all units to be deleted.

Silhouette prevents a unit with patients assigned to it from being deleted.



## Close

Select **Close** to go back to the **Unit Administration** screen.

## SilhouetteStar 2


The SilhouetteStar 2 settings available depend on if you are using the camera in wired USB mode with SilhouetteConnect or in Wi-Fi mode with SilhouetteCentral.

- Wired mode with SilhouetteConnect has options for SilhouetteStar 2 DHCP Configuration.
- Wi-Fi mode with SilhouetteCentral has options for Network Configurations and Wireless Camera Tools.

## SilhouetteStar 2 DHCP Configuration

When operating with SilhouetteConnect in wired USB mode, the SilhouetteStar 2 establishes a network connection with the computer using an Internet Protocol (IP) address. The SilhouetteStar 2 camera provides the computer an IP address via the Dynamic Host Configuration Protocol (DHCP). The default IP addressused by the camera is 192.168.7.2 /24 and it assigns the address 192.168.7.1 /24 to the computer. In some rare cases the network used may cause a conflict with other networks used in the organization and the SilhouetteStar 2 will not be able to establish a connection to SilhouetteConnect.

The **Admin > SilhouetteStar 2** page provides SilhouetteStar 2 DHCP Configuration settings. These settings are available in SilhouetteConnect in both synchronized and standalone operational modes.

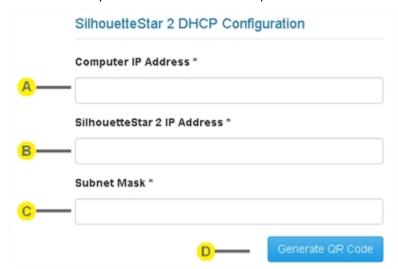



## **Computer IP Address**

Enter the IPv4 Address (in dotted decimal notation) that the SilhouetteStar 2 camera should be assigned to the computer. This address must be in the same IPv4 subnet as the SilhouetteStar 2 IP Address, with respect to the Subnet Mask.

The default factory setting is 192.168.7.1



## SilhouetteStar 2 IP Address

Enter the IPv4 Address (in dotted decimal notation) for the SilhouetteStar 2 camera. This address must be in the same IPv4 subnet as the Computer IP Address, with respect to the Subnet Mask.

The default factory setting is 192.168.7.2



#### **Subnet Mask**

Enter the IPv4 Subnet Mask (in dotted decimal notation) for the SilhouetteStar 2 / Computer link network. This Subnet Mask must allow the network to span the Computer IP Address and the SilhouetteStar 2 IP Address.

The default factory setting is 255.255.255.0



#### **Generate QR Code**

Once valid IP addresses have been entered, click on the **Generate QR Code** button. A QR Code encoding the settings is displayed on the screen.



#### Apply the Settings to Camera

To apply the settings to the SilhouetteStar 2 camera:

- With the SilhouetteStar 2 camera turned on and not connected to Silhouette,
- Press the camera button to turn on the lasers (begins scanning for QR Code),
- Hold the SilhouetteStar 2 up to the screen with the button at the top and so that the lasers form a star over the QR code,
- The camera will display "DHCP Updated" once the QR Code has been scanned and the settings have been applied.


Once these settings are applied to a camera, they are remembered until the camera firmware is updated.

The QR code can be printed and used to set all your cameras to the same settings. There is no need to generate a new QR code each time. The QR code in the example diagram above can be used to set the camera back to default settings.

#### Checking the SilhouetteStar 2 DHCP Configuration

If you wish to confirm the settings the camera is using then you can check the settings once the camera is plugged into a PC by:

- Navigating to the Control Panel > Network and Internet > Network Connections.
- Right-click on the Ethernet Adapter that has a SilhouetteStar 2 label and selecting Status.
- On the Ethernet Adapter status page click the **Details...** button.
- The IPv4 Address, IPv4 Subnet Mask and IPv4 DHCP Server are all listed in the Network Connection Details dialog and correspond to the Computer IP Address, the Subnet Mask, and the SilhouetteStar 2 IP Address settings in Silhouette.

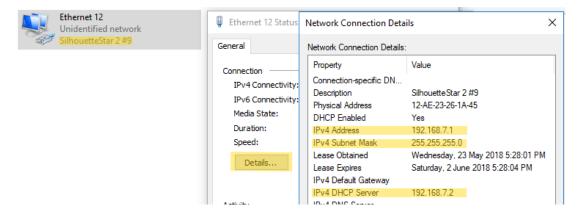


# **Network Configurations and Wireless Camera Tools**

The settings in the **SilhouetteStar 2** screen are used to allow the SilhouetteStar 2 camera to connect to the organization's Wi-Fi network.

These settings are displayed within the user application through a QR code which the camera uses to read the network settings and successfully connect.

To define and manage network configurations or to access the wireless camera tools, select **Admin > SilhouetteStar**2 in SilhouetteCentral. The **Network Configurations** and **Wireless Camera Tools** sections are not available in SilhouetteConnect.

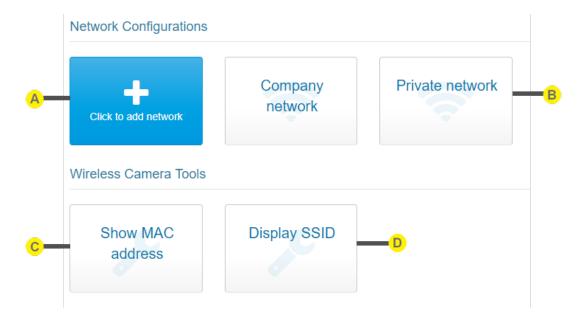



#### **Create button**

See "Creating a Network Configuration".



### **Network Configuration tile**

See "Editing a Network Configuration".



#### **Show MAC Address**

Select Show MAC address to navigate to a QR code that can be scanned by a SilhouetteStar 2 camera to show the Wi-Fi MAC address of the camera on the camera display.

Once scanned, the camera shows the Wi-Fi MAC address until it is dismissed with a camera button press.



#### **Display SSID**

Select Display SSID to navigate to a QR code that can be scanned by a SilhouetteStar 2 camera to cause the network SSID to be displayed on the camera display.

The SSID is shown on the third line of the display, replacing the Patient DoB. The SSID of the network is displayed whenever the camera is connected, or is attempting to connect, to a Wi-Fi network.



The SilhouetteStar 2 continues show the SSID on the display until it is power cycled or until the Display SSID QR code is scanned again.




The Wireless Camera Tools QR codes can only be scanned if the camera is not already connected to a network.



🌎 The Wireless Camera Tools QR codes can be printed, or downloaded as images and emailed. This may help if you need Silhouette users without admin access to scan them for diagnostic or setup purposes.

### **Creating a Network Configuration**


Select the Click to add network button in the SilhouetteStar 2 screen to create a new network configuration. The Create Network screen is displayed, with compulsory fields marked with an asterisk. Enter a Name, Type, Network SSID, and Network Password.

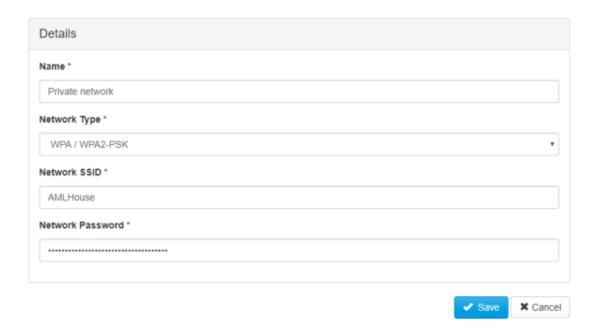

To save the network's settings, select the **Save** button at the bottom of the screen.

## **Editing a Network Configuration**

To edit a network configuration, in the **SilhouetteStar 2** screen, select the network from the networks list. The information for the network is displayed in a read-only view.

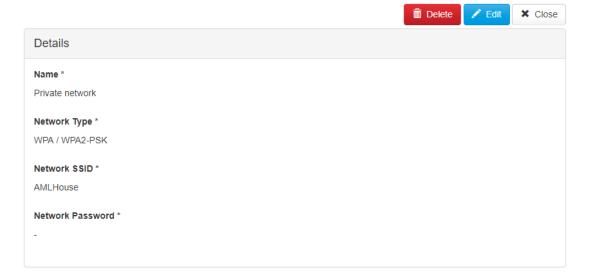

Select the **Edit** button. Update the information, as required. To save the network's settings, select the **Save** button at the bottom of the screen.

To delete the group, use the **Delete** button.

Select **Close** to go back to the Network Configurations and Wireless Camera Tools screen.

# **System Requirements and Operating Conditions**

This section may not list the full set of system specifications. A full system specification sheet can be obtained from ARANZ Medical.

# SilhouetteCentral Web Application - Minimum Requirements

Web Browsers Supported Safari, IE, Edge, Firefox, Chrome

Note: The web application is not designed to work on phone screens. An

iPad mini or larger is recommended.

## SilhouetteConnect - Minimum Requirements

Operating System Windows 10

Windows 7 and Windows 8.1 may be supported where extended support

is in place. Contact ARANZ medical for verification.

Processor 2.0 GHz (Multi-core recommended)

Memory 2 GB RAM (4 GB Recommended)

Network Ethernet or Wi-Fi TCP/IP Connection required for synchronization to

Silhouette Central

Hard Disk 10 GB Available Space

The required space depends on how many images and assessments are needed to be held offline. The 10 GB space allowance is sufficient for approximately 5000 images. Each image can vary in space requirements

depending on the complexity of the image contents.

Database Included in SilhouetteConnect (SQL Compact Edition)

The maximum database size is approximately 250 MB. This typically

equates to 30000 to 40000 assessments for offline patients.

Display Resolution 1024 x 768 (Minimum)

### SilhouetteStar 2 - Minimum Requirements

Wi-Fi Network Support Wi-Fi 802.11 a,b,g at 2.4 GHz

Wi-Fi Security Supported WPA/WPA2-PSK

Note: Network passwords must be at least 8 characters long

Note: 802.1x Authentication with MAC Authentication Bypass can also be used with a WPA2-PSK. This configuration requires site testing to confirm

operation with site access points and authentication servers.

Minimum bandwidth required 14 Mbps uplink and downlink between the exam room and the

SilhouetteCentral server.

Maximum network latency 300 ms round trip between the exam room and the SilhouetteCentral

server.

Lighting 100 lux minimum

400 lux nominal (normal office) 1000 lux max (bright office)

Operating Temperature 10 C (50 F) min

30 C (86 F) max

Operating Humidity 0% RH min

80% RH max

Storage Temperature -5 C (23 F) min

50 C (122 F) max

The SilhouetteStar 2 camera uses Wi-Fi to send captured images to SilhouetteCentral. The Wi-Fi network at the point of care (i.e. in the exam rooms) must be reliable to provide dependable camera operation. A reasonable heuristic for the Wi-Fi performance required is that the Wi-Fi should support a 10 minute video conference session without glitches and with good video quality.

The bandwidth and latency requirements above are minimum recommendations for end-to-end network performance requirements based on testing. They do not cater for the number of other services using the network or packet loss in the network. As part of the deployment it is recommended that some testing be carried out to confirm reliable and timely performance of the system.

# SilhouetteLite and SilhouetteLite+ App - Minimum Requirements

iOS version iOS 11 or newer

Device support iPhone 5s and later

The application interface is designed primarily for iPhone Plus and iPad Mini devices but has been tested on a wider range of device sizes.

#### 38 • SilhouetteLite+ Sensor - Operating Conditions

Security Passcode / Touch ID enabled

It is recommended that passcode / Touch ID is enabled on the device. Doing so enables the Apple device to encrypt the SilhouetteLite files,

protecting them from untrusted access.

Maximum number of patients 4000

Maximum number of images per

image capture field

10

# SilhouetteLite+ Sensor - Operating Conditions

x offset -12 mm to +20 mm

Horizontal offset for placement of the SilhouetteLite+ sensor optics

relative to the device optics.

y offset -12 mm to +30 mm

Vertical offset for placement of the SilhouetteLite+ sensor optics relative

to the device optics.

Sensor Operating Temperature 0°C to 40°C (32°F to 104°F)

Sensor Charging Temperature 0°C to 45°C (32°F to 113°F)

Sensor Storage Temperature  $-5^{\circ}$ C to  $60^{\circ}$ C (23°F to 140°F) for < 2 hours

 $0^{\circ}$ C to  $45^{\circ}$ C ( $32^{\circ}$ F to  $113^{\circ}$ F) for up to 1 month

0°C to 35°C (32°F to 95°F) for > 1 month

# **Appendices**

The appendices include:







Appendix B: Anatomical Sites



**Appendix C: Notes Definitions** 



Appendix D: Regular Expressions

# **Appendix A: Getting Support**

You can contact ARANZ Medical support by using the contact information listed below:

### **Customer Support**

Email: support@aranzmedical.com

Phone (International): +64 3 961 1988

Phone (US Customers): +1 877 274 5468

Please contact your system administrator for first level

support contact information.

### **Sales Enquiries**

Phone (International): +64-3-374-6120

Phone (US customers): 866-467-0934

Fax: +64-3-374-6130

Email: sales@aranzmedical.com

### **ARANZ Medical Website**

www.aranzmedical.com

# **Appendix B: Anatomical Sites**

Wounds recorded within SilhouetteConnect and SilhouetteCentral are always assigned to an anatomical site for reporting and identification purposes.

When specifying an anatomical site the user is prompted to select options from a series of hierarchical drop-down lists. The options available in the next drop-down list depend upon the selection the user made in the previous.

The following tables outline the available anatomical sites and the selections a user must make in the Gross Division, Handedness and Location drop-down lists in order to select them.

The anatomical sites described in this appendix reflect the default set of anatomical sites that SilhouetteCentral and SilhouetteConnect are setup with. They do not show any customisations made.

### **Head & Neck**

| Gross Division | Handedness | Location      | Display within Silhouette |
|----------------|------------|---------------|---------------------------|
| Scalp          |            | Superior      | Scalp, Superior           |
|                |            | Posterior     | Scalp, Posterior          |
|                |            | Left Lateral  | Scalp, Left Lateral       |
|                |            | Right Lateral | Scalp, Right Lateral      |
| Forehead       |            | Anterior      | Forehead, Anterior        |
|                |            | Left Lateral  | Forehead, Left Lateral    |
|                |            | Right Lateral | Forehead, Right Lateral   |
| Temple         | Left       |               | Left Temple               |
|                | Right      |               | Right Temple              |
| Orbit          | Left       | Superior      | Left Orbit, Superior      |
|                |            | Inferior      | Left Orbit, Inferior      |
|                |            | Eyeball       | Left Orbit, Eyeball       |
|                |            | Medial        | Left Orbit, Medial        |
|                |            | Lateral       | Left Orbit, Lateral       |
|                | Right      | Superior      | Right Orbit, Superior     |
|                |            | Inferior      | Right Orbit, Inferior     |
|                |            | Eyeball       | Right Orbit, Eyeball      |
|                |            | Medial        | Right Orbit, Medial       |
|                |            | Lateral       | Right Orbit, Lateral      |

| Cheek | Left  | Superior      | Left Cheek, Superior   |
|-------|-------|---------------|------------------------|
|       |       |               |                        |
|       |       | Inferior      | Left Cheek, Inferior   |
|       |       | Anterior      | Left Cheek, Anterior   |
|       |       | Posterior     | Left Cheek, Posterior  |
|       |       | Central       | Left Cheek             |
|       | Right | Superior      | Right Cheek, Superior  |
|       |       | Inferior      | Right Cheek, Inferior  |
|       |       | Anterior      | Right Cheek, Anterior  |
|       |       | Posterior     | Right Cheek, Posterior |
|       |       | Central       | Right Cheek            |
| Ear   | Left  | Superior      | Left Ear, Superior     |
|       |       | Inferior      | Left Ear, Inferior     |
|       |       | Anterior      | Left Ear, Anterior     |
|       |       | Posterior     | Left Ear, Posterior    |
|       | Right | Superior      | Right Ear, Superior    |
|       |       | Inferior      | Right Ear, Inferior    |
|       |       | Anterior      | Right Ear, Anterior    |
|       |       | Posterior     | Right Ear, Posterior   |
| Nose  |       | Superior      | Nose, Superior         |
|       |       | Inferior      | Nose, Inferior         |
|       |       | Anterior      | Nose, Anterior         |
|       |       | Left Lateral  | Nose, Left Lateral     |
|       |       | Right Lateral | Nose, Right Lateral    |
| Lip   |       | Upper         | Lip, Upper             |
|       |       | Lower         | Lip, Lower             |
|       |       | Left Angle    | Lip, Left Angle        |
|       |       | Right Angle   | Lip, Right Angle       |
| Jaw   |       | Anterior      | Jaw, Anterior          |
|       |       | Inferior      | Jaw, Inferior          |
|       |       |               | Jaw, Left Lateral      |
|       |       | Right Lateral | Jaw, Right Lateral     |

| Gross Division | Handedness | Location      | Display within Silhouette |
|----------------|------------|---------------|---------------------------|
| Neck           | ,          | Anterior      | Neck, Anterior            |
|                |            | Left Lateral  | Neck, Left Lateral        |
|                |            | Right Lateral | Neck, Right Lateral       |
|                |            | Posterior     | Neck, Posterior           |

# Torso

| Gross Division | Handedness | Location          | Display within Silhouette    |
|----------------|------------|-------------------|------------------------------|
| Thorax         | Left       | Clavicular region | Left Clavicular region       |
|                |            | Axilla            | Left Axilla                  |
|                |            | Lateral region    | Left Lateral region, Thorax  |
|                | Right      | Clavicular region | Right Clavicular region      |
|                |            | Axilla            | Right Axilla                 |
|                |            | Lateral region    | Right Lateral region, Thorax |
| Sternum        |            | Superior          | Sternum, Superior            |
|                |            | Inferior          | Sternum, Inferior            |
| Breast         | Left       | Superior          | Left Breast, Superior        |
|                |            | Inferior          | Left Breast, Inferior        |
|                |            | Anterior          | Left Breast, Anterior        |
|                |            | Medial            | Left Breast, Medial          |
|                |            | Lateral           | Left Breast, Lateral         |
|                |            | Nipple            | Left Breast, Nipple          |
|                | Right      | Superior          | Right Breast, Superior       |
|                |            | Inferior          | Right Breast, Inferior       |
|                |            | Anterior          | Right Breast, Anterior       |
|                |            | Medial            | Right Breast, Medial         |
|                |            | Lateral           | Right Breast, Lateral        |
|                |            | Nipple            | Right Breast, Nipple         |

| Gross Division | Handedness | Location             | Display within Silhouette     |
|----------------|------------|----------------------|-------------------------------|
| Abdomen        |            | Right upper quadrant | Abdomen, Right upper quadrant |
|                |            | Right lower quadrant | Abdomen, Right lower quadrant |
|                |            | Left upper quadrant  | Abdomen, Left upper quadrant  |
|                |            | Left lower quadrant  | Abdomen, Left lower quadrant  |
|                |            | Midline - upper      | Abdomen, Midline - upper      |
|                |            | Umbilical            | Abdomen, Umbilical            |
|                |            | Midline - lower      | Abdomen, Midline - lower      |
|                |            | Left flank           | Abdomen, Left flank           |
|                |            | Right flank          | Abdomen, Right flank          |
| Back           | Left       | Scapular region      | Left Back, Scapular region    |
|                |            | Thoracic region      | Left Back, Thoracic region    |
|                |            | Lumbar region        | Left Back, Lumbar region      |
|                | Right      | Scapular region      | Right Back, Scapular region   |
|                |            | Thoracic region      | Right Back, Thoracic region   |
|                |            | Lumbar region        | Right Back, Lumbar region     |
| Spine          | •          | Thoracic             | Spine, Thoracic               |
|                |            | Lumbar               | Spine, Lumbar                 |

# **Pelvis**

| Gross Division      | Handedness | Location  | Display within Silhouette |
|---------------------|------------|-----------|---------------------------|
| Spine/Gluteal cleft |            | Sacrum    | Sacrum                    |
|                     |            | Соссух    | Соссух                    |
|                     |            | Peri-anal | Peri-anal                 |
| Buttock             | Left       | Superior  | Left Buttock, Superior    |
|                     |            | Inferior  | Left Buttock, Inferior    |
|                     |            | Central   | Left Buttock              |
|                     |            | Medial    | Left Buttock, Medial      |
|                     |            | Lateral   | Left Buttock, Lateral     |
|                     | Right      | Superior  | Right Buttock, Superior   |
|                     |            | Inferior  | Right Buttock, Inferior   |
|                     |            | Central   | Right Buttock             |
|                     |            | Medial    | Right Buttock, Medial     |
|                     |            | Lateral   | Right Buttock, Lateral    |

| Gross Division         | Handedness | Location           | Display within Silhouette |
|------------------------|------------|--------------------|---------------------------|
| Misc. bony prominences | Left       | Iliac Crest        | Left Iliac Crest          |
|                        |            | Greater trochanter | Left Trochanter           |
|                        |            | Ischial Tuberosity | Left Ischial Tuberosity   |
|                        | Right      | Iliac Crest        | Right Iliac Crest         |
|                        |            | Greater trochanter | Right Trochanter          |
|                        |            | Ischial Tuberosity | Right Ischial Tuberosity  |
| Groin                  | Left       |                    | Left Groin                |
|                        | Right      |                    | Right Groin               |
| Genital Male           |            | Scrotum            | Scrotum                   |
|                        |            | Penis              | Penis                     |
|                        |            | Perineum           | Perineum                  |
| Genital Female         |            | Vulva              | Vulva                     |
|                        |            | Labia              | Labia                     |
|                        |            | Perineum           | Perineum                  |

# **Upper Extremity**

| Gross Division | Handedness | Location  | Display within Silhouette |
|----------------|------------|-----------|---------------------------|
| Shoulder       | Left       | Superior  | Left Shoulder, Superior   |
|                |            | Anterior  | Left Shoulder, Anterior   |
|                |            | Posterior | Left Shoulder, Posterior  |
|                |            | Lateral   | Left Shoulder, Lateral    |
|                | Right      | Superior  | Right Shoulder, Superior  |
|                |            | Anterior  | Right Shoulder, Anterior  |
|                |            | Posterior | Right Shoulder, Posterior |
|                |            | Lateral   | Right Shoulder, Lateral   |

## 46 • Upper Extremity

| Gross Division | Handedness | Location        | Display within Silhouette        |
|----------------|------------|-----------------|----------------------------------|
| Upper Arm      | Left       | Anterior        | Left Upper Arm, Anterior         |
|                |            | Posterior       | Left Upper Arm, Posterior        |
|                |            | Medial          | Left Upper Arm, Medial           |
|                |            | Lateral         | Left Upper Arm, Lateral          |
|                |            | Circumferential | Left Upper Arm, Circumferential  |
|                | Right      | Anterior        | Right Upper Arm, Anterior        |
|                |            | Posterior       | Right Upper Arm, Posterior       |
|                |            | Medial          | Right Upper Arm, Medial          |
|                |            | Lateral         | Right Upper Arm, Lateral         |
|                |            | Circumferential | Right Upper Arm, Circumferential |
| Elbow          | Left       | Anterior        | Left Elbow, Anterior             |
|                |            | Posterior       | Left Elbow, Posterior            |
|                |            | Medial          | Left Elbow, Medial               |
|                |            | Lateral         | Left Elbow, Lateral              |
|                |            | Circumferential | Left Elbow, Circumferential      |
|                | Right      | Anterior        | Right Elbow, Anterior            |
|                |            | Posterior       | Right Elbow, Posterior           |
|                |            | Medial          | Right Elbow, Medial              |
|                |            | Lateral         | Right Elbow, Lateral             |
|                |            | Circumferential | Right Elbow, Circumferential     |
| Forearm        | Left       | Anterior        | Left Forearm, Anterior           |
|                |            | Posterior       | Left Forearm, Posterior          |
|                |            | Medial          | Left Forearm, Medial             |
|                |            | Lateral         | Left Forearm, Lateral            |
|                |            | Circumferential | Left Forearm, Circumferential    |
|                | Right      | Anterior        | Right Forearm, Anterior          |
|                |            | Posterior       | Right Forearm, Posterior         |
|                |            | Medial          | Right Forearm, Medial            |
|                |            | Lateral         | Right Forearm, Lateral           |
|                |            | Circumferential | Right Forearm, Circumferential   |

| <b>Gross Division</b> | Handedness | Location        | Display within Silhouette    |
|-----------------------|------------|-----------------|------------------------------|
| Wrist                 | Left       | Anterior        | Left Wrist, Anterior         |
|                       |            | Posterior       | Left Wrist, Posterior        |
|                       |            | Medial          | Left Wrist, Medial           |
|                       |            | Lateral         | Left Wrist, Lateral          |
|                       |            | Circumferential | Left Wrist, Circumferential  |
|                       | Right      | Anterior        | Right Wrist, Anterior        |
|                       |            | Posterior       | Right Wrist, Posterior       |
|                       |            | Medial          | Right Wrist, Medial          |
|                       |            | Lateral         | Right Wrist, Lateral         |
|                       |            | Circumferential | Right Wrist, Circumferential |
| Hand                  | Left       | Palm            | Left Hand, Palm              |
|                       |            | Dorsum          | Left Hand, Dorsum            |
|                       |            | Medial          | Left Hand, Medial            |
|                       |            | Lateral         | Left Hand, Lateral           |
|                       |            | Circumferential | Left Hand, Circumferential   |
|                       | Right      | Palm            | Right Hand, Palm             |
|                       |            | Dorsum          | Right Hand, Dorsum           |
|                       |            | Medial          | Right Hand, Medial           |
|                       |            | Lateral         | Right Hand, Lateral          |
|                       |            | Circumferential | Right Hand, Circumferential  |
| Finger                | Left       | Thumb           | Left Thumb                   |
|                       |            | Index Finger    | Left Index Finger            |
|                       |            | Middle Finger   | Left Middle Finger           |
|                       |            | Ring Finger     | Left Ring Finger             |
|                       |            | Little Finger   | Left Little Finger           |
|                       | Right      | Thumb           | Right Thumb                  |
|                       |            | Index Finger    | Right Index Finger           |
|                       |            | Middle Finger   | Right Middle Finger          |
|                       |            | Ring Finger     | Right Ring Finger            |
|                       |            | Little Finger   | Right Little Finger          |

# **Lower Extremity**

| Gross Division | Handedness | Location        | Display within Silhouette        |
|----------------|------------|-----------------|----------------------------------|
| Thigh          | Left       | Anterior        | Left Thigh, Anterior             |
|                |            | Posterior       | Left Thigh, Posterior            |
|                |            | Medial          | Left Thigh, Medial               |
|                |            | Lateral         | Left Thigh, Lateral              |
|                |            | Circumferential | Left Thigh, Circumferential      |
|                | Right      | Anterior        | Right Thigh, Anterior            |
|                |            | Posterior       | Right Thigh, Posterior           |
|                |            | Medial          | Right Thigh, Medial              |
|                |            | Lateral         | Right Thigh, Lateral             |
|                |            | Circumferential | Right Thigh, Circumferential     |
| Knee           | Left       | Anterior        | Left Knee, Anterior              |
|                |            | Posterior       | Left Knee, Posterior             |
|                |            | Medial          | Left Knee, Medial                |
|                |            | Lateral         | Left Knee, Lateral               |
|                |            | Circumferential | Left Knee, Circumferential       |
|                | Right      | Anterior        | Right Knee, Anterior             |
|                |            | Posterior       | Right Knee, Posterior            |
|                |            | Medial          | Right Knee, Medial               |
|                |            | Lateral         | Right Knee, Lateral              |
|                |            | Circumferential | Right Knee, Circumferential      |
| Lower leg      | Left       | Shin            | Left Shin                        |
|                |            | Calf            | Left Calf                        |
|                |            | Medial          | Lower left leg, Medial           |
|                |            | Lateral         | Lower left leg, Lateral          |
|                |            | Circumferential | Lower left leg, Circumferential  |
|                | Right      | Shin            | Right Shin                       |
|                |            | Calf            | Right Calf                       |
|                |            | Medial          | Lower right leg, Medial          |
|                |            | Lateral         | Lower right leg, Lateral         |
|                |            | Circumferential | Lower right leg, Circumferential |

| Gross Division | Handedness | Location          | Display within Silhouette |
|----------------|------------|-------------------|---------------------------|
| Ankle          | Left       | Medial Malleolus  | Left Medial Malleolus     |
|                |            | Lateral Malleolus | Left Lateral Malleolus    |
|                |            | Ankle Flexure     | Left Ankle Flexure        |
|                |            | Posterior         | Left Ankle, Posterior     |
|                | Right      | Medial Malleolus  | Right Medial Malleolus    |
|                |            | Lateral Malleolus | Right Lateral Malleolus   |
|                |            | Ankle Flexure     | Right Ankle Flexure       |
|                |            | Posterior         | Right Ankle, Posterior    |
| Foot           | Left       | Dorsum            | Left Foot, Dorsum         |
|                |            | Sole              | Left Foot, Sole           |
|                |            | Medial            | Left Foot, Medial         |
|                |            | Lateral           | Left Foot, Lateral        |
|                |            | Heel              | Left Foot, Heel           |
|                | Right      | Dorsum            | Right Foot, Dorsum        |
|                |            | Sole              | Right Foot, Sole          |
|                |            | Medial            | Right Foot, Medial        |
|                |            | Lateral           | Right Foot, Lateral       |
|                |            | Heel              | Right Foot, Heel          |
| Toe            | Left       | Big               | Left Toe, Big             |
|                |            | Second            | Left Toe, Second          |
|                |            | Third             | Left Toe, Third           |
|                |            | Fourth            | Left Toe, Fourth          |
|                |            | Fifth             | Left Toe, Fifth           |
|                | Right      | Big               | Right Toe, Big            |
|                |            | Second            | Right Toe, Second         |
|                |            | Third             | Right Toe, Third          |
|                |            | Fourth            | Right Toe, Fourth         |
|                |            | Fifth             | Right Toe, Fifth          |

# **Appendix C: Notes Definitions**

In addition to images and measurements SilhouetteConnect and SilhouetteCentral can be used to capture notes. Notes can be associated with a particular patient, visit or wound assessment. The following section of this document outlines the note fields available for clinicians to enter.

The notes described in this appendix reflect the default set of notes that SilhouetteCentral and SilhouetteConnect are setup with. They do not show any customisations made.

### **Patient admission**

### **Details**

| Item               | Variable                | Data Type | Visibility |     |
|--------------------|-------------------------|-----------|------------|-----|
| Expected Discharge | expected_discharge_date | Date      | Active     | Yes |
|                    |                         |           | Required   | No  |
|                    |                         |           | Visible    | Yes |
| Palliative Care    | palliative_care         | Boolean   | Active     | Yes |
|                    |                         |           | Required   | No  |
|                    |                         |           | Visible    | Yes |

# **Physicians**

| Item      | Variable                                        | Data Type                                 | Visibility                    |                  |
|-----------|-------------------------------------------------|-------------------------------------------|-------------------------------|------------------|
| Treating  | admission_and_discharge_<br>treating_physician  | Text  • Max Length: 255  • Multi-Line: No | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Referring | admission_and_discharge_<br>referring_physician | Text  • Max Length: 255  • Multi-Line: No | Active<br>Required<br>Visible | Yes<br>No<br>Yes |

# **Patient Details**

## **Details**

| Item          | Variable              | Data Type                                 | Visibility                    |                   |
|---------------|-----------------------|-------------------------------------------|-------------------------------|-------------------|
| Patient ID    | details_patient_id    | Text  • Max Length: 128                   | Active<br>Required            | Yes<br>Yes        |
|               |                       | Multi-Line: No                            | Visible                       | Yes               |
| Unit          | details_unit          | Unit                                      | Active<br>Required<br>Visible | Yes<br>Yes<br>Yes |
| Last Name     | details_last_name     | Text  • Max Length: 128  • Multi-Line: No | Active<br>Required<br>Visible | Yes<br>Yes<br>Yes |
| First Name    | details_first_name    | Text  • Max Length: 128  • Multi-Line: No | Active<br>Required<br>Visible | Yes<br>Yes<br>Yes |
| Middle Name   | details_middle_name   | Text  • Max Length: 128  • Multi-Line: No | Active<br>Required<br>Visible | Yes<br>No<br>Yes  |
| Date of Birth | details_date_of_birth | Date                                      | Active<br>Required<br>Visible | Yes<br>Yes<br>Yes |
| Gender        | details_is_female     | List (single selection)                   | Active<br>Required<br>Visible | Yes<br>Yes<br>Yes |
|               | Male     Female       | 1                                         | ı                             |                   |

# Address

| Item     | Variable         | Data Type       | Visibility  |
|----------|------------------|-----------------|-------------|
| Street   | address_street   | Text            | Active Yes  |
|          |                  | Max Length: 255 | Required No |
|          |                  | Multi-Line: No  | Visible Yes |
| Suburb   | address_suburb   | Text            | Active Yes  |
|          |                  | Max Length: 255 | Required No |
|          |                  | Multi-Line: No  | Visible Yes |
| City     | address_city     | Text            | Active Yes  |
|          |                  | Max Length: 255 | Required No |
|          |                  | Multi-Line: No  | Visible Yes |
| State    | address_state    | Text            | Active Yes  |
|          |                  | Max Length: 255 | Required No |
|          |                  | Multi-Line: No  | Visible Yes |
| Postcode | address_postcode | Text            | Active Yes  |
|          |                  | Max Length: 50  | Required No |
|          |                  | Multi-Line: No  | Visible Yes |
| Country  | address_country  | Text            | Active Yes  |
|          |                  | Max Length: 255 | Required No |
|          |                  | Multi-Line: No  | Visible Yes |

# **Contact**

| Item         | Variable             | Data Type      | Visibility |     |
|--------------|----------------------|----------------|------------|-----|
| Home Phone   | contact_home_phone   | Text           | Active     | Yes |
|              |                      | Max Length: 50 | Required   | No  |
|              |                      | Multi-Line: No | Visible    | Yes |
| Mobile Phone | contact_mobile_phone | Text           | Active     | Yes |
|              |                      | Max Length: 50 | Required   | No  |
|              |                      | Multi-Line: No | Visible    | Yes |
| Work Phone   | contact_work_phone   | Text           | Active     | Yes |
|              |                      | Max Length: 50 | Required   | No  |
|              |                      | Multi-Line: No | Visible    | Yes |

# **Patient discharge**

## **Details**

| Item     | Variable                                                                | Data Type                                  | Visibility                    |                  |
|----------|-------------------------------------------------------------------------|--------------------------------------------|-------------------------------|------------------|
| Reason   | admission_status                                                        | List (single<br>selection)                 | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|          | <ul><li>Discharged</li><li>Lost to follow-up</li><li>Deceased</li></ul> |                                            |                               |                  |
| Comments | discharge_comments                                                      | Text  • Max Length: 255  • Multi-Line: Yes | Active<br>Required<br>Visible | Yes<br>No<br>Yes |

# **Assessment of lower limb function**

# Right

| Item   | Variable                                                     | Data Type                  | Visibility                         |
|--------|--------------------------------------------------------------|----------------------------|------------------------------------|
| Warmth | lower_limb_assessment_<br>warmth_right_leg                   | List (single<br>selection) | Active Yes Required No Visible Yes |
|        | <ul><li>Cold</li><li>Cool</li><li>Warm</li><li>Hot</li></ul> |                            |                                    |

| Item                      | Variable                                                                                                                                                | Data Type                    | Visibility                    |                  |
|---------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------|-------------------------------|------------------|
| Venous signs              | lower_limb_assessment_<br>venous_signs_right_leg                                                                                                        | List (multiple<br>selection) | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                           | <ul> <li>Varicose veins</li> <li>Atrophie blanche</li> <li>Eczema</li> <li>Lipodermatosclerosis</li> <li>Hemosiderin staining</li> </ul>                |                              |                               |                  |
| Calf muscle effectiveness | lower_limb_assessment_calf_<br>muscle_effectiveness_right                                                                                               | Boolean                      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Arterial signs            | lower_limb_assessment_<br>arterial_signs_right_leg                                                                                                      | List (multiple<br>selection) | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                           | <ul> <li>Hairless</li> <li>Shiny</li> <li>Capillary refill &gt; 3 seconds</li> <li>Dependent rubor</li> <li>Atrophic nails</li> <li>Cyanosis</li> </ul> |                              |                               |                  |
| Pulses palpable           | lower_limb_assessment_<br>pulses_palpable_right                                                                                                         | List (multiple<br>selection) | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                           | <ul><li>Dorsalis pedis</li><li>Posterior tibial</li></ul>                                                                                               |                              |                               |                  |
| Dorsalis pedis pressure   | lower_limb_assessment_<br>dorsalis_pedis_pressure_right                                                                                                 | Decimal                      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |

| Item                      | Variable                                                        | Data Type                    | Visibility                    |                  |
|---------------------------|-----------------------------------------------------------------|------------------------------|-------------------------------|------------------|
| Posterior tibial pressure | lower_limb_assessment_<br>posterior_tibial_pressure_right       | Decimal                      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Brachial pressure         | lower_limb_assessment_<br>brachial_pressure_right               | Decimal                      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Pulse signal              | lower_limb_assessment_<br>pulse_signal_right                    | List (multiple<br>selection) | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                           | <ul><li>Monophasic</li><li>Biphasic</li><li>Triphasic</li></ul> |                              |                               |                  |
| ABPI result               | lower_limb_assessment_abpi_<br>result_right                     | Decimal                      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Toe pressure              | lower_limb_assessment_toe_<br>pressure_right                    | Decimal                      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| TBPI                      | lower_limb_assessment_tbpi_<br>right                            | Decimal                      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Thinnest ankle            | lower_limb_assessment_<br>thinnest_ankle_right                  | Decimal (cm)                 | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Widest calf               | lower_limb_assessment_<br>widest_calf_right                     | Decimal (cm)                 | Active<br>Required<br>Visible | Yes<br>No<br>Yes |

# **Ambulation**

| Item                                        | Variable                                                                                                                         | Data Type               | Visibility                    |                  |
|---------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------|-------------------------|-------------------------------|------------------|
| Ambulation                                  | lower_limb_assessment_<br>ambulation                                                                                             | List (single selection) | Active<br>Required            | Yes<br>No        |
|                                             |                                                                                                                                  |                         | Visible                       | Yes              |
|                                             | <ul> <li>Independent - no aid or assist</li> <li>Independent with mobility at</li> <li>Chair bound</li> <li>Bed bound</li> </ul> |                         |                               |                  |
| Client report of walking duration (minutes) | lower_limb_assessment_<br>client_report_of_walking_<br>duration_minutes                                                          | Decimal                 | Active<br>Required<br>Visible | Yes<br>No<br>Yes |

# Left

| Item         | Variable                                                                                                                                 | Data Type                    | Visibility                    |                  |
|--------------|------------------------------------------------------------------------------------------------------------------------------------------|------------------------------|-------------------------------|------------------|
| Warmth       | lower_limb_assessment_<br>warmth_left_leg                                                                                                | List (single selection)      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|              | <ul><li>Cold</li><li>Cool</li><li>Warm</li><li>Hot</li></ul>                                                                             |                              |                               |                  |
| Venous signs | lower_limb_assessment_<br>venous_signs_left_leg                                                                                          | List (multiple<br>selection) | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|              | <ul> <li>Varicose veins</li> <li>Atrophie blanche</li> <li>Eczema</li> <li>Lipodermatosclerosis</li> <li>Hemosiderin staining</li> </ul> |                              |                               |                  |

| Item                      | Variable                                                                                                                                                | Data Type                    | Visibility                    |                  |
|---------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------|-------------------------------|------------------|
| Calf muscle effectiveness | lower_limb_assessment_calf_<br>muscle_effectiveness_left                                                                                                | Boolean                      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Arterial signs            | lower_limb_assessment_<br>arterial_signs_left_leg                                                                                                       | List (multiple<br>selection) | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                           | <ul> <li>Hairless</li> <li>Shiny</li> <li>Capillary refill &gt; 3 seconds</li> <li>Dependent rubor</li> <li>Atrophic nails</li> <li>Cyanosis</li> </ul> |                              |                               |                  |
| Pulses palpable           | lower_limb_assessment_<br>pulses_palpable_left                                                                                                          | List (multiple<br>selection) | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                           | <ul><li>Dorsalis pedis</li><li>Posterior tibial</li></ul>                                                                                               |                              | ,                             |                  |
| Dorsalis pedis pressure   | lower_limb_assessment_<br>dorsalis_pedis_pressure_left                                                                                                  | Decimal                      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Posterior tibial pressure | lower_limb_assessment_<br>posterior_tibial_pressure_left                                                                                                | Decimal                      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Brachial pressure         | lower_limb_assessment_<br>brachial_pressure_left                                                                                                        | Decimal                      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |

## 58 • Comments

| Item           | Variable                                                           | Data Type                    | Visibility                    |                  |
|----------------|--------------------------------------------------------------------|------------------------------|-------------------------------|------------------|
| Pulse signal   | lower_limb_assessment_<br>pulse_signal_left                        | List (multiple<br>selection) | Active<br>Required            | Yes<br>No        |
|                | <ul><li> Monophasic</li><li> Biphasic</li><li> Triphasic</li></ul> |                              | Visible                       | Yes              |
| ABPI result    | lower_limb_assessment_abpi_<br>result_left                         | Decimal                      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Toe pressure   | lower_limb_assessment_toe_<br>pressure_left                        | Decimal                      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| ТВРІ           | lower_limb_assessment_tbpi_<br>left                                | Decimal                      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Thinnest ankle | lower_limb_assessment_<br>thinnest_ankle_left                      | Decimal (cm)                 | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Widest calf    | lower_limb_assessment_<br>widest_calf_left                         | Decimal (cm)                 | Active<br>Required<br>Visible | Yes<br>No<br>Yes |

## **Comments**

| Item     | Variable                           | Data Type                                | Visibility                    |                  |
|----------|------------------------------------|------------------------------------------|-------------------------------|------------------|
| Comments | lower_limb_assessment_<br>comments | Text • Max Length: 255 • Multi-Line: Yes | Active<br>Required<br>Visible | Yes<br>No<br>Yes |

# **Visit Assessment**

# Vital signs

| Item                      | Variable                     | Data Type        | Visibility |     |
|---------------------------|------------------------------|------------------|------------|-----|
| Weight                    | vital_signs_weight           | Decimal (kg)     | Active     | Yes |
|                           |                              |                  | Required   | No  |
|                           |                              |                  | Visible    | Yes |
| Height                    | vital_signs_height           | Decimal (cm)     | Active     | Yes |
|                           |                              |                  | Required   | No  |
|                           |                              |                  | Visible    | Yes |
| Pulse rate                | vital_signs_pulse_rate       | Decimal (/min)   | Active     | Yes |
|                           |                              |                  | Required   | No  |
|                           |                              |                  | Visible    | Yes |
| Respiration rate          | vital_signs_respiration_rate | Decimal (/min)   | Active     | Yes |
|                           |                              |                  | Required   | No  |
|                           |                              |                  | Visible    | Yes |
| Blood pressure, systolic  | vital_signs_blood_pressure_  | Decimal (mmHg)   | Active     | Yes |
|                           | systolic                     |                  | Required   | No  |
|                           |                              |                  | Visible    | Yes |
| Blood pressure, diastolic | vital_signs_blood_pressure_  | Decimal (mmHg)   | Active     | Yes |
|                           | diastolic                    |                  | Required   | No  |
|                           |                              |                  | Visible    | Yes |
| Temperature               | vital_signs_temperature      | Decimal ( C)     | Active     | Yes |
|                           |                              |                  | Required   | No  |
|                           |                              |                  | Visible    | Yes |
| Blood glucose level       | vital_signs_blood_glucose_   | Decimal (mmol/L) | Active     | Yes |
|                           | level                        |                  | Required   | No  |
|                           |                              |                  | Visible    | Yes |
|                           | I                            | 1                | 1          |     |

# **Related Pain (Non-Wound)**

| Item         | Variable                                                                                                                               | Data Type                                  | Visibility                    |                  |
|--------------|----------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------|-------------------------------|------------------|
| Location     | related_pain_nonwound_<br>location                                                                                                     | Text  • Max Length: 255  • Multi-Line: No  | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Scale (0-10) | related_pain_nonwound_<br>scale_010                                                                                                    | Integer Interval: 1 Range: 0 to 10         | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Occurrence   | related_pain_nonwound_<br>occurrence                                                                                                   |                                            | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|              | <ul> <li>At dressing change</li> <li>Night pain</li> <li>On walking</li> <li>On elevation</li> <li>Dependent</li> <li>Other</li> </ul> |                                            |                               |                  |
| Comments     | related_pain_nonwound_<br>comments                                                                                                     | Text  • Max Length: 255  • Multi-Line: Yes | Active<br>Required<br>Visible | Yes<br>No<br>Yes |

# Edema

| Item    | Variable      | Data Type | Visibility |     |
|---------|---------------|-----------|------------|-----|
| Pitting | edema_pitting | Boolean   | Active     | Yes |
|         |               |           | Required   | No  |
|         |               |           | Visible    | Yes |

| Item             | Variable             | Data Type       | Visibility |     |
|------------------|----------------------|-----------------|------------|-----|
| Location (right) | edema_location_right | List (multiple  | Active     | Yes |
|                  |                      | selection)      | Required   | No  |
|                  |                      |                 | Visible    | Yes |
|                  | • Toes               |                 |            |     |
|                  | • Foot               |                 |            |     |
|                  | Gaiter               |                 |            |     |
|                  | Above gaiter         |                 |            |     |
|                  | • Knee               |                 |            |     |
|                  | • Thigh              |                 |            |     |
| Location (left)  | edema_location_left  | List (multiple  | Active     | Yes |
|                  |                      | selection)      | Required   | No  |
|                  |                      |                 | Visible    | Yes |
|                  | • Toes               |                 |            |     |
|                  | • Foot               |                 |            |     |
|                  | • Gaiter             |                 |            |     |
|                  | Above gaiter         |                 |            |     |
|                  | • Knee               |                 |            |     |
|                  | • Thigh              |                 |            |     |
| Location: Other  | edema_location_other | Text            | Active     | Yes |
|                  |                      | Max Length: 255 | Required   | No  |
|                  |                      | Multi-Line: No  | Visible    | Yes |
| Comments         | edema_comments       | Text            | Active     | Yes |
|                  |                      | Max Length: 255 | Required   | No  |
|                  |                      | Multi-Line: Yes | Visible    | Yes |

# **Factors Affecting Healing**

| Item                      | Variable                                              | Data Type | Visibility |                |
|---------------------------|-------------------------------------------------------|-----------|------------|----------------|
| Impaired sensation (left) | factors_affecting_healing_<br>impaired_sensation_left | Boolean   | Required N | es<br>lo<br>es |

## 62 • Factors Affecting Healing

| Item                       | Variable                                                                             | Data Type               | Visibility                    |                  |
|----------------------------|--------------------------------------------------------------------------------------|-------------------------|-------------------------------|------------------|
| Impaired sensation (right) | factors_affecting_healing_<br>impaired_sensation_right                               | Boolean                 | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Braden outcome             | factors_affecting_healing_<br>braden_outcome                                         | List (single selection) | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                            | <ul><li>Low risk</li><li>Mod risk</li><li>High risk</li><li>Very high risk</li></ul> |                         |                               |                  |
| PURA outcome               | factors_affecting_healing_<br>pura_outcome                                           | List (single selection) | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                            | <ul><li>Low risk</li><li>Mod risk</li><li>High risk</li><li>Very high risk</li></ul> |                         |                               |                  |

| Item                    | Variable                                                                                                                                                                                                                                                                                                                                                                                                                                           | Data Type                 | Visibility                    |                  |
|-------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------|-------------------------------|------------------|
| Nutritional Risk screen | factors_affecting_healing_<br>nutritional_risk_screen                                                                                                                                                                                                                                                                                                                                                                                              | List (multiple selection) | Active                        | Yes              |
|                         | Tiddidonal_risk_screen                                                                                                                                                                                                                                                                                                                                                                                                                             | selection)                | Required                      | No               |
|                         |                                                                                                                                                                                                                                                                                                                                                                                                                                                    |                           | Visible                       | Yes              |
|                         | <ul> <li>Obvious underweight-frailty</li> <li>Unintentional weight loss</li> <li>Reduced appetite or food and fluid intake</li> <li>Mouth teeth problems</li> <li>Swallowing problems</li> <li>Follows a special diet</li> <li>Needs assistance to shop for food</li> <li>Needs assistance to prepare food</li> <li>Needs assistance to feed self</li> <li>Obvious overweight affecting life quality</li> <li>Unintentional weight gain</li> </ul> |                           |                               |                  |
| BMI outcome             | factors_affecting_healing_<br>bmi_outcome                                                                                                                                                                                                                                                                                                                                                                                                          | List (single selection)   | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                         | <ul> <li>Underweight (&lt;18.5)</li> <li>Normal (18.5 - 24.9)</li> <li>Overweight (25 - 29.9)</li> <li>Obese (&gt;30)</li> </ul>                                                                                                                                                                                                                                                                                                                   |                           |                               |                  |

# **Wound Assessment**

# **Images**

| Item         | Variable                        | Data Type     | Visibility                    |                  |
|--------------|---------------------------------|---------------|-------------------------------|------------------|
| Wound Images | images_wound_images             | Image Capture | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|              | Trace images screen is enabled. |               |                               |                  |

| 64 |  |  |  |
|----|--|--|--|
|    |  |  |  |
|    |  |  |  |

**Details** 

| Item     | Variable               | Data Type               | Visibility |     |
|----------|------------------------|-------------------------|------------|-----|
| Etiology | wound_history_etiology | List (single selection) | Active     | Yes |
|          |                        | Selection               | Required   | No  |
|          |                        |                         | Visible    | Yes |

| Item | Variable                 | Data Type                 | Visibility |
|------|--------------------------|---------------------------|------------|
|      | Pressure Ulcer: Stage    | 1                         |            |
|      | Pressure Ulcer: Stage    | 2                         |            |
|      | Pressure Ulcer: Stage    | 3                         |            |
|      | Pressure Ulcer: Stage    | 4                         |            |
|      | Pressure Ulcer: Unsta    | ageable (dressing/device) |            |
|      | Pressure Ulcer: Unsta    | ageable (eschar/slough)   |            |
|      | Deep Tissue Injury       |                           |            |
|      | Venous Ulcer             |                           |            |
|      | Arterial insufficiency   |                           |            |
|      | Mixed Venous/Arteria     | al                        |            |
|      | • Diabetic               |                           |            |
|      | Neuropathic: Non-dia     | abetic                    |            |
|      | Surgical: Closed         |                           |            |
|      | Surgical: Full thicknes  | S                         |            |
|      | Surgical: Partial thickr | ness                      |            |
|      | Surgical: Dehiscence     |                           |            |
|      | Drainage Device          |                           |            |
|      | Trauma: Full thickness   | S                         |            |
|      | Trauma: Partial thickn   | ness                      |            |
|      | Trauma: Superficial      |                           |            |
|      | Skin tear: Category 1    |                           |            |
|      | Skin tear: Category 2a   | 1                         |            |
|      | Skin tear: Category 2b   | )                         |            |
|      | Skin tear: Category 3    |                           |            |
|      | Burn: Superficial thick  | kness                     |            |
|      | Burn: Partial thickness  | S                         |            |
|      | Burn: Full thickness     |                           |            |
|      | Cancerous: Fungating     | lesion                    |            |
|      | Cancerous: Ulcerating    | g lesion                  |            |
|      | Cancerous: Other         |                           |            |
|      | Skin graft               |                           |            |
|      | • Donor site             |                           |            |
|      | Pilonidal wound          |                           |            |

| Item             | Variable                                      | Data Type                 | Visibility |     |
|------------------|-----------------------------------------------|---------------------------|------------|-----|
| Wound Margins    | wound_margins_sloping                         | List (multiple            | Active     | Yes |
|                  |                                               | selection)                | Required   | No  |
|                  |                                               |                           | Visible    | Yes |
|                  | <ul><li>Sloping</li><li>Punched out</li></ul> |                           |            |     |
|                  | Rolled                                        |                           |            |     |
|                  | Everted                                       |                           |            |     |
|                  | <ul> <li>Undermining</li> </ul>               |                           |            |     |
|                  | • Sinus                                       |                           |            |     |
|                  | • Inflamed                                    |                           |            |     |
| Surrounding Skin | surrounding_skin_tissue_                      | List (multiple selection) | Active     | Yes |
|                  | paper_skin                                    |                           | Required   | No  |
|                  |                                               |                           | Visible    | Yes |
|                  | 'Tissue paper' skin                           |                           |            |     |
|                  | Peri-wound edema                              |                           |            |     |
|                  | Macerated                                     |                           |            |     |
|                  | • Erythema                                    |                           |            |     |
|                  | <ul> <li>Inflammation</li> </ul>              |                           |            |     |
|                  | • Pustules                                    |                           |            |     |
|                  | • Eczema                                      |                           |            |     |
|                  | • Dry/scaly                                   |                           |            |     |
|                  | Healthy                                       |                           |            |     |

| Item                                           | Variable                                                                                                                                                                                                                                                            | Data Type                  | Visibility                    |                  |
|------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------|-------------------------------|------------------|
| Signs of Critical Colonization or<br>Infection | signs_of_critical_colonization_<br>or_infection_cellulitis                                                                                                                                                                                                          | List (multiple selection)  | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                                                | <ul> <li>Cellulitis</li> <li>Suppuration</li> <li>Lymphangitis</li> <li>Sepsis</li> <li>Bacteremia</li> <li>Granulation changes</li> <li>Exudate increase</li> <li>Increase/new pain</li> <li>Impaired/delayed healing</li> <li>Wound breakdown/new slow</li> </ul> | ugh                        |                               |                  |
| Exudate Volume                                 | exudate_volume                                                                                                                                                                                                                                                      | List (single selection)    | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                                                | <ul><li>None</li><li>Low</li><li>Moderate</li><li>High</li></ul>                                                                                                                                                                                                    |                            |                               |                  |
| Exudate Type                                   | exudate_type                                                                                                                                                                                                                                                        | List (single<br>selection) | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                                                | <ul><li>Serous</li><li>Serosanguineous</li><li>Sanguineous</li><li>Purulent</li></ul>                                                                                                                                                                               |                            |                               |                  |

# **Wound Pain**

| Item           | Variable                         | Data Type         | Visibility |     |  |  |  |  |
|----------------|----------------------------------|-------------------|------------|-----|--|--|--|--|
| Frequency      | wound_pain_frequency             | List (single      | Active     | Yes |  |  |  |  |
|                |                                  | selection)        | Required   | No  |  |  |  |  |
|                |                                  |                   | Visible    | Yes |  |  |  |  |
|                | • Absent                         |                   |            |     |  |  |  |  |
|                | <ul> <li>Intermittent</li> </ul> |                   |            |     |  |  |  |  |
|                | • Continuous                     |                   |            |     |  |  |  |  |
| Nature of pain | wound_pain_nature_of_pain        | List (multiple    | Active     | Yes |  |  |  |  |
|                |                                  | selection)        | Required   | No  |  |  |  |  |
|                |                                  |                   | Visible    | Yes |  |  |  |  |
|                | • Throbbing                      |                   |            |     |  |  |  |  |
|                | • Shooting                       | • Shooting        |            |     |  |  |  |  |
|                | Stabbing                         | Stabbing          |            |     |  |  |  |  |
|                | • Sharp                          | • Sharp           |            |     |  |  |  |  |
|                | Cramping                         | Cramping          |            |     |  |  |  |  |
|                | Gnawing                          |                   |            |     |  |  |  |  |
|                | Hot/burning                      | Hot/burning       |            |     |  |  |  |  |
|                | • Aching                         |                   |            |     |  |  |  |  |
|                | • Heavy                          |                   |            |     |  |  |  |  |
|                | • Tender                         |                   |            |     |  |  |  |  |
|                | Splitting                        |                   |            |     |  |  |  |  |
|                | Tiring/exhausting                | Tiring/exhausting |            |     |  |  |  |  |
|                | Sickening                        |                   |            |     |  |  |  |  |
|                | Fearful                          |                   |            |     |  |  |  |  |
|                | Cruel/punishing                  |                   |            |     |  |  |  |  |
|                | • Other                          |                   |            |     |  |  |  |  |

| Item                      | Variable                                                                                                       | Data Type                                  | Visibility                    |                  |
|---------------------------|----------------------------------------------------------------------------------------------------------------|--------------------------------------------|-------------------------------|------------------|
| Current interventions     | wound_pain_current_<br>interventions                                                                           | List (multiple<br>selection)               | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                           | <ul><li>Pharmacological</li><li>Non-pharmacological</li><li>Dressing/removal technique</li><li>Other</li></ul> |                                            |                               |                  |
| At dressing change (0-10) | wound_pain_at_dressing_<br>change_010                                                                          | Integer Interval: 1 Range: 0 to 10         | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Night pain (0-10)         | wound_pain_night_pain_010                                                                                      | Integer Interval: 1 Range: 0 to 10         | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| On walking (0-10)         | wound_pain_on_walking_010                                                                                      | Integer Interval: 1 Range: 0 to 10         | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| On elevation (0-10)       | wound_pain_on_elevation_<br>010                                                                                | Integer Interval: 1 Range: 0 to 10         | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Dependent (0-10)          | wound_pain_dependent_010                                                                                       | Integer Interval: 1 Range: 0 to 10         | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Other (0-10)              | wound_pain_other_010                                                                                           | Integer Interval: 1 Range: 0 to 10         | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Comments                  | wound_pain_other_<br>description                                                                               | Text  • Max Length: 255  • Multi-Line: Yes | Active<br>Required<br>Visible | Yes<br>No<br>Yes |

# Comments

| Item     | Variable            | Data Type       | Visibility |    |
|----------|---------------------|-----------------|------------|----|
| Comments | assessment_comments | Text            | Active Y   | es |
|          |                     | Max Length: 255 | Required N | lo |
|          |                     | Multi-Line: Yes | Visible Y  | es |

# **Investigation History**

# **Investigation History**

| Item       | Variable                                                                                                                                | Data Type                                  | Visibility                    |                  |
|------------|-----------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------|-------------------------------|------------------|
| Diagnostic | investigation_history_<br>diagnostic                                                                                                    | List (multiple selection)                  | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|            | <ul><li>Venous duplex</li><li>Arterial duplex</li><li>Angiogram</li><li>X-ray</li></ul>                                                 |                                            |                               |                  |
| Pathology  | investigation_history_<br>pathology                                                                                                     | List (multiple selection)                  | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|            | <ul> <li>Wound biopsy</li> <li>Hb</li> <li>Thyroid function</li> <li>Albumin</li> <li>Creatinine</li> <li>HbA1c</li> <li>FBC</li> </ul> |                                            |                               |                  |
| Comments   | investigation_history_<br>comments                                                                                                      | Text  • Max Length: 255  • Multi-Line: Yes | Active<br>Required<br>Visible | Yes<br>No<br>Yes |

# **Medical History**

# **Medical History**

| Item               | Variable                                                                                                                                                                                                                                                                                                                                    | Data Type                    | Visibility                    |                  |
|--------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------|-------------------------------|------------------|
| Current conditions | medical_history_current_<br>conditions                                                                                                                                                                                                                                                                                                      | List (multiple<br>selection) | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                    | <ul> <li>Diabetes</li> <li>Cardiac condition</li> <li>Anemia</li> <li>COAD</li> <li>Rheumatoid arthritis</li> <li>Autoimmune disorder</li> <li>Hepatic failure</li> <li>Malignancy</li> <li>Peripheral vascular disease</li> <li>Neurological disorder</li> <li>Inflammatory bowel disease</li> <li>Renal disease</li> <li>Other</li> </ul> |                              |                               |                  |

| Item                          | Variable                                                                                                                                                                                                                                                                                 | Data Type                    | Visibility                    |                  |
|-------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------|-------------------------------|------------------|
| Contributing factors          | medical_history_contributing_<br>factors                                                                                                                                                                                                                                                 | List (multiple<br>selection) | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                               | <ul> <li>Reduced mobility</li> <li>Pressure</li> <li>Shearing</li> <li>Friction</li> <li>Impaired sensation</li> <li>Edema</li> <li>Nutritional factors</li> <li>Obesity</li> <li>Radiotherapy</li> <li>Smoking</li> <li>Venous hypertension</li> <li>Ischemia</li> <li>Other</li> </ul> |                              |                               |                  |
| Smoking History               | smoking_history                                                                                                                                                                                                                                                                          | List (single selection)      | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                               | <ul><li>Smoker</li><li>Past smoker</li><li>Never smoked</li></ul>                                                                                                                                                                                                                        | 1                            |                               |                  |
| Current cigarette consumption | current_cigarette_<br>consumption                                                                                                                                                                                                                                                        | List (single<br>selection)   | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|                               | <ul><li>1-10 per day</li><li>11-20 per day</li><li>21 or more per day</li></ul>                                                                                                                                                                                                          |                              |                               |                  |

## 74 • Medication

| Item                 | Variable                                 | Data Type                                  | Visibility                    |                  |
|----------------------|------------------------------------------|--------------------------------------------|-------------------------------|------------------|
| Past medical history | medical_history_past_<br>medical_history | Text  • Max Length: 255  • Multi-Line: Yes | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
| Comments             | medical_history_comments                 | Text  • Max Length: 255  • Multi-Line: Yes | Active<br>Required<br>Visible | Yes<br>No<br>Yes |

# Medication

## Medication

| Item      | Variable                                                                                                                                                                          | Data Type                                  | Visibility                    |                  |
|-----------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------|-------------------------------|------------------|
| Medicines | medication_medicines                                                                                                                                                              | List (multiple<br>selection)               | Active<br>Required<br>Visible | Yes<br>No<br>Yes |
|           | <ul> <li>NSAIDS</li> <li>Cytotoxics</li> <li>Anti-coagulants</li> <li>Beta blockers</li> <li>Immune suppressants</li> <li>Steroids</li> <li>Antibiotics</li> <li>Other</li> </ul> |                                            |                               |                  |
| Comments  | medication_comments                                                                                                                                                               | Text  • Max Length: 255  • Multi-Line: Yes | Active<br>Required<br>Visible | Yes<br>No<br>Yes |

## **Sensitivities**

| Item      | Variable                               | Data Type                                 | Visibility                         |
|-----------|----------------------------------------|-------------------------------------------|------------------------------------|
| Medicines | medication_sensitivities_<br>medicines | Text  • Max Length: 255  • Multi-Line: No | Active Yes Required No Visible Yes |

| Item                    | Variable                                             | Data Type       | Visibility |     |
|-------------------------|------------------------------------------------------|-----------------|------------|-----|
| Dressings and adhesives | medication_sensitivities_<br>dressings_and_adhesives | Text            | Active     | Yes |
|                         |                                                      | Max Length: 255 | Required   | No  |
|                         |                                                      | Multi-Line: No  | Visible    | Yes |
| Other                   | medication_sensitivities_other                       | Text            | Active     | Yes |
|                         |                                                      | Max Length: 255 | Required   | No  |
|                         |                                                      | Multi-Line: No  | Visible    | Yes |

# **Appendix D: Regular Expressions**

Regular Expressions are a tool which allow a description of how a piece of information should be structured, for example a phone number or email address. This allows computer software to process the information reliably as it conforms to a pre-defined pattern.

**Note:** For the examples that follow, square brackets [] are used to specify the range of characters required to be used in that group and curly brackets {} to specify the quantity of characters that are required.

## **Patient Identifier Examples for Silhouette**

### **Example A**

A 5 digit patient number would be written as:

- [0-9]{5}
- Values satisfying this regular expression include 12345 and 75319

### **Example B**

A 3 digit unit ID, a dash, then a 3 digit patient number would be written as:

- [0-9]{3}-[0-9]{3}
- Values satisfying this regular expression include 147-963 and 456-321

### **Example C**

Two upper-case alphabetic character unit ID, then 4 digit patient number would be written as:

- [A-Z]{2}[0-9]{4}
- Values satisfying this regular expression include ZK1648 and VT7642

# **Example D**

A three-digit unit ID, a dash, a patient number comprising 4, 5 or 6 numeric digits, and the last character being an upper-case A, B, or C to indicate three different study groups would be written as:

- [0-9]{3}-[0-9]{4,6}[ABC]{1}
- Values satisfying this regular expression include 010-01934A and 110-50012C

Through the simple use of regular expressions you can control the accuracy of information entered into a system and hence improve reliability and reduce errors.

Further reading and an interactive tutorial can be found here: http://regexone.com/

### Other examples

- International phone numbers should begin with a plus "+" character e.g. +64 for New Zealand.
- An email address should contain only one "@" character
- · A bank account number should begin with a two-digit bank identification number
- A credit card number comprises four groups of four digits
- A financial amount can comprise numeric digits, decimal point, commas, and the dollar sign e.g. \$41,031.64
- A patient name comprising first name, optional middle name, and surname.

Using Regular Expressions the following types of structures can be specified:

- A character from a group of characters, e.g.:
  - A lower-case alphabetic character [a-z]
  - An upper-or-lower case alphabetic character [a-zA-Z]
  - A numeric digit [0-9]
  - Alphanumeric characters comprising lower-case alphabetic or numeric digits [0-9a-z]
  - Commonly occurring punctuation characters such as dash "-", comma ",", period ".", brackets "[", and braces "(", etc.
- A character can occur a fixed number of times, e.g.:
   Three characters which can be either a, b or c would be written as: [abc]{3}
- A single optional character is indicated by the question mark "?", e.g.:
   An optional uppercase alphabetic character would be written as: [A-Z]?
- 1 upper-case alphabetic or 2 numeric digits could be required using the special or operator "|":
   such as: [A-Z]|[0-9]{2}

Some real-world examples include:

- A credit card number four groups of four digits separated by a dash: [0-9]{4}-[0-9]{4}-[0-9]{4}
- A New Zealand national phone number a two digit area code in brackets, followed by three digits, then a dash, then four digits:

\(0[1-9]\)[0-9]{3}-[0-9]{4}

### **Things to Note**

• The special treatment of the "(" and ")" characters —these have special meaning within Regular Expression language so when they need to be used as normal characters this needs to be indicated. Preceding special characters with the "\" ("slash") character tells the computer to treat the character as regular text - this is referred to as "escaping" a character. Other characters that may need escaping include "[", "]", even the slash

## 78 • Things to Note

itself requires escaping using a slash. "\\".

• The dash "-" character generally doesn't require escaping, although it does have special meaning when used to designate a character range within the square brackets. If a dash is used outside a character range statement then it will be interpreted as a normal character.